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1. Product's name:

1.1. Technical Name: Self-ligating Ceramic Brackets

1.2. Commercial name: Aditek Self-ligating Ceramic Brackets

1.3. Commercial Model: EasyClip Clear Ceramic Brackets, Prescriptions: Roth, MBT, Andrews, Damon Low Torque, Damon Standard, Damon Super Torque, Armando Low Torque (L), Armando Medium (M), Armando High Torque (H); Active Clear Ceramic Brackets, Prescriptions: Roth, MBT, Andrews, Damon Low Torque, Damon Standard, Damon Super Torque; BioClip Clear Ceramic Brackets, Prescriptions: Roth, MBT, Andrews, Damon Low Torque, Damon Standard, Damon Super Torque, Voudouris, Two-dimensional, Two-dimensional Hybrid; EasyClip PI Clear Ceramic Brackets, Prescriptions: Roth, MBT, Andrews, Damon Low Torque, Damon Standard, Damon Super Torque, Voudouris, Two-dimensional, Two-dimensional Hybrid; BioClip PI Clear Ceramic Brackets, Prescriptions: Roth, MBT, Andrews, Damon Low Torque, Damon Standard, Damon Super Torque, Voudouris, Two-dimensional, Two-dimensional Hybrid; Affinity Clear Ceramic Brackets, Prescriptions: Roth, MBT, Andrews, Damon Low Torque, Damon Standard, Damon Super Torque, Armando Low Torque (L), Armando Medium (M), Armando High Torque (H); Select Clear Ceramic Brackets, Prescriptions: Roth, MBT, Damon Low Torque, Damon Standard, Damon Super Torque, Standard I, Standard II, Standard III; Evolution Clear Ceramic Brackets, Prescriptions: Roth, MBT, Standard I, Standard II, Standard III; Variflex Clear Ceramic Brackets, Prescriptions: Roth, MBT, Damon Low Torque, Damon Standard, Damon Super Torque; Stilo 3D Clear Ceramic Brackets, Prescription: Lingual; Ceramic Bracket Variflex Clear Ceramic Brackets, Prescriptions: Roth, MBT, Damon Low Torque, Damon Standard, Damon Super Torque; Stilo 3D Clear Ceramic Brackets, Prescription: Lingual; Ceramic Bracket Variflex Clear Ceramic Brackets, Prescriptions: Roth, MBT, Damon Low Torque, Damon Standard, Damon Super Torque; Stilo 3D Clear Ceramic Brackets, Prescription: Lingual; Ceramic BracketSlide H5 Clear, Prescriptions: Roth, MBT, Damon Low Torque, Damon Standard, Damon Super Torque.

1.4. Anvisa Notification:10331430027

2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Mota, 14.

CEP: 14140-000 - Cravinhos - SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Arantes Nogueira Neto - CREA: 601037885

www.aditek.com.br

3. Certifying entity:

Not applicable

4. Usage Indication:

Aditek Self-ligating Ceramic Brackets are indicated as auxiliary components in orthodontic treatment. The purpose of using brackets is to serve as a point of support and fitting for wires, preformed arches or other auxiliary products used in conventional orthodontic treatments.

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The products were developed to be used in the oral cavity environment, directly on the teeth, with the objective of supporting orthodontic treatments in human beings, promoting a better positioning of the teeth within the dental arches, seeking a balanced occlusion.

Brackets are used in conjunction with arches, wires, springs, bands, elastics and other auxiliary products necessary for orthodontic treatment.

They are indicated to treat patients who have malocclusion problems that require alignment and leveling movements of the teeth in the arches.

They are chosen by orthodontic professionals who seek a more aesthetic and faster clinical care in the exchange of wires or orthodontic arches.

The brackets have a clip that keeps the wire inside the slot, eliminating the need for ligatures or tying wires.

The Slide H5 was designed to be used with light strength arches, with thermal memory, with a maximum caliber of up to .017”x.025”, according to the recommendations presented in the table below:

| Arc Sequence Recommendation | | |
|---|---|------------------|
| Phase I | Alignment Phase | arc deflection |
| <ul style="list-style-type: none"> • Alignment and leveling • arcade development | .013” Contour NiTi .014” Contour NiTi or Heat Activated NiTi | More than 0.5mm |
| Phase II | Leveling Phase | |
| <ul style="list-style-type: none"> • complete rotation • torque control • angulation correction • Flatten Curve of Spee | .016” Contour NiTi or Thermoactivated NiTi .014”x.025” Contour NiTi .016”x.025” Contour NiTi .016”x.022” Nitinol Reverse Curve | 0.35mm to 0.5mm |
| Phase III | Work Phase | |
| <ul style="list-style-type: none"> • Closing spaces • Anterior retraction with torque control | .016”x.022” Thermoactivated NiTi or .016”x.025” Contour NiTi | Less than 0.35mm |
| Phase IV | Finishing Phase | |
| <ul style="list-style-type: none"> • midline correction • Root alignment and final torque control | .016”x.022” TMA .017”x.025” Thermoactivated NiTi | Less than 0.15mm |

NOTE:The choice of material to be used is made by the qualified professional, based on their knowledge acquired during their training together with the diagnosis and treatment plan for each patient.

5. Users:

The application of Slide H5 brackets is restricted to dental professionals. The use by people outside this context may cause harm to the patient.

6. Performance Features:

As the brackets are glued directly to the surface of the teeth, all the force applied to it is transferred directly to the tooth causing its position to be changed and consequently allowing its correction to the desired correct position.

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Commonly, an uninterrupted orthodontic treatment takes from 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion and oral and systemic conditions of the patient, and can be extended according to the complexity of the case.

7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are the aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

8. Warnings:

 Aditek products are designed and manufactured for single use only. Its reuse or reprocessing can cause cross-infection, decrease in gluing efficiency and loss of mechanical properties due to natural wear and forces applied to product removal.

 If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

The product must be applied by a dentist who must follow the instructions for use and guidelines specified by the manufacturer.

The product is safe for use in pregnant women. It is known that due to hormonal changes during pregnancy, women are more susceptible to periodontal problems. During orthodontic treatment, difficulty in cleaning can exacerbate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

The procedure for bonding and removing the brackets on the teeth must be performed according to the instructions for use of the manufacturer of the orthodontic adhesive purchased.

The composition of the product does not require specific procedures for its transport and does not present risks to people.

The product is supplied in a non-sterile condition, however it has proven biocompatibility, requiring no cleaning, disinfection or sterilization measures, it is up to the professional to perform the sterilization.

The professional must use good clinical practices for the disposal of medical products according to current health regulations.

Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

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In the event of serious incidents with the product, Aditek do Brasil must be notified to the competent authority at the national level in which the dentist is established.

9. Contraindications:

- Patients with poor oral hygiene;
- Patients with inability to cooperate with treatment;
- Illness or limitation that may make treatment difficult;
- Existing bone or root resorption;
- Decalcification of existing tooth enamel;
- Existing periodontal complications;
- More complex treatments that require extensive biomechanics.

NOTE: Aditek do Brasil S.A. supplies products only to dental professionals. It is the sole responsibility of the dentist identification and/or possible contraindication that may prevent the use of the product.

10. Residual Risks:

All finished orthodontic treatments are subject to relapse. After treatment, control must be determined by the professional in order to avoid or minimize relapse.

11. Unwanted Side Effects:

The use of the medical device can lead to discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort and pain.

12. Composition:

| |
|--|
| Non-Toxic Polycrystalline Alumina Ceramic |
| Chemical composition limits (%) |
| 100% |

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14. Symbols and Meanings:

| | | | |
|---|------------------------------|---|---|
|  | Manufacturer |  | Do not reuse the product |
|  | Consult instructions for use |  | Do not use if package is damaged or tampered with |
|  | Heads up |  | Expiration date |
|  | Product reference code |  | Lot number |

15. Patient Information:

The patient must perform good daily hygiene in order to avoid the installation of bacterial plaque and consequent periodontal diseases and to remove food particles that may settle under the clip. The use of an oral irrigator, type Water Pik, is recommended to help clean teeth and brackets.

It is important that the pregnant patient is referred for professional cleaning at regular intervals, due to the increased susceptibility to periodontal problems during pregnancy.

It is important to be careful when chewing certain types of food, especially hard or crunchy ones, as they can come off or break the brackets. These foods must be cut before eating. Hard foods like popcorn and ice should be avoided.

Do not put objects in your mouth, such as pens and pencils, and be careful not to hit or bite the toothbrush structure when brushing your teeth, which can break or loosen the bracket when subjected to a prying movement.

Foods and/or drinks with a lot of coloring, natural or not, can stain the brackets. When drinking beverages such as wine, coffee, açai, soft drinks, rinse your mouth with water after ingesting them;

Smoking or chewing tobacco can stain your brackets.

In the event of a malfunction, seek a dentist.

In case of ingestion or aspiration, seek medical attention.

When participating in sports, always wear mouth guards, as impacts can damage the appliances and injure the oral cavity.

The patient must follow the professional's guidelines regarding containment to avoid relapse.

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
16. Usage Mode:


The steps below should only be performed by dental professionals.

Prepare the surface of the teeth according to the instructions for use of the manufacturer of the orthodontic adhesive purchased.

Glue the brackets to the teeth according to the instructions for use of the manufacturer of the orthodontic adhesive purchased (We suggest the use of the LightBond Aditek adhesive)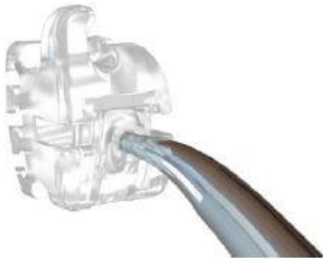


Place the fine-tipped instrument in the appropriate housing in the center of the clip;

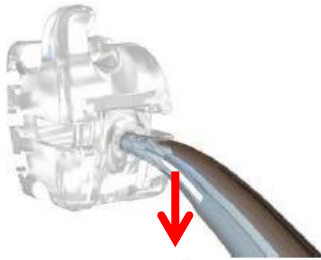Gently pull the instrument vertically and occlusally. Excessive force may cause the clip to fall;

Position the selected arch according to the treatment phase to be performed;

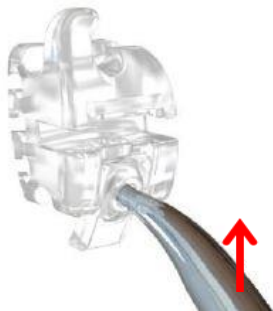To close, position the instrument and move the clip towards cervical. It is important to feel the clip lock into the bulge and hear the locking click.

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16.1. Bracket Removal:

Remove the resin around the bracket with a high-speed resin removal cutter.

Position the bracket removing instrument in the mesiodistal direction and make a clockwise movement. The bracket must come loose, leaving a small portion of resin on the tooth, which must be removed with a high-speed resin removal cutter or resin removing pliers.

NOTE: If you have difficulty removing the bracket, with a diamond bur, section the bracket horizontally, at the height of the slot, taking care not to touch the dental enamel. After sectioning, remove the parts with a suitable instrument.

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