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1. Product's name:

1.1. Technical Name: Orthodontic Bracket

1.2. Commercial name: Aditek Ceramic Brackets

1.3. Commercial Model: bracket Ceramic translux.018 Standard I; Ceramic Bracket Translux .022 Pattern I; Ceramic Bracket Translux .018 Pattern II; Ceramic Bracket Translux .022 Pattern II; Ceramic Bracket Translux .018 Standard III; Ceramic Bracket Translux .022 Standard III; Ceramic Bracket Translux .018 Andrews; Ceramic Bracket Translux .022 Andrews; Ceramic Bracket Translux .018 Roth; Ceramic Bracket Translux .022 Roth; Ceramic Bracket Translux Mini .018 Roth; Ceramic Bracket Translux Mini .022 Roth; Ceramic Bracket Translux .018 MBT; Ceramic Bracket Translux .022 MBT; Ceramic Bracket Translux Mini .018 MBT; Ceramic Bracket Translux Mini .022 MBT; Ceramic Bracket Translux .018 Ricketts; Ceramic Bracket Translux .022 Ricketts; Ceramic Bracket Translux .018 Edgewise; Ceramic Bracket Translux .022 Edgewise; Ceramic Bracket Translux Mini .018 Edgewise; Ceramic Bracket Translux Mini .Sapphire bracket.018 Roth; Bracket BioTru Sapphire .018 Roth; . Sapphire Bracket .022 Roth; Bracket BioTru Sapphire .022 Roth Bracket Sapphire .018 MBT; BioTru Sapphire Bracket .018 MBT; Sapphire Bracket .022 MBT; BioTru Sapphire Bracket .022 MBT; Sapphire Bracket .018 Edgewise; Bracket BioTru Sapphire .018 Edgewise; Sapphire Bracket .022 Edgewise; Bracket BioTru Sapphire .022 Edgewise; Clear Tube Single Non-Convertible Roth Superior Glue; Clear Tube Single Non-Convertible Glue Superior MBT; Clear Tube Single Non-Convertible Glue Superior Damon; Clear Tube Single Non-Convertible Glue Top/Bottom, Right/Left Edgewise; Clear Tube Single Non-Convertible Glue Bottom Roth; Clear Tube Single Non-Convertible Glue Bottom MBT; Clear Tube Single Non-Convertible Glue Bottom Damon; Mini Clear Tube Simple Non-Convertible Glue 2nd Upper Molar Roth; Mini Clear Tube Simple Non-Convertible Glue 2nd Upper Molar MBT; Mini Clear Tube Simple Non-Convertible Glue 2nd Upper Molar Damon; Mini Clear Tube Simple Non-Convertible Glue 2nd Molar Upper/Lower Edgewise; Mini Clear Tube Simple Non-Convertible Glue 2nd Lower Molar Roth; Tube Mini Clear Simple Non-Convertible Glue 2nd Lower Molar MBT; Mini Clear Tube Simple Non-Convertible Glue 2nd Lower Molar Damon; Mini Clear Button For Glue; Bite Turbo Clear; Ceramic Bracket Translux+ .018 Roth; Bracket BioTru Ceramic .018 Roth; Ceramic Bracket Translux+ .022 Roth; Bracket BioTru Ceramic .022 Roth; Ceramic Bracket Translux+ .018 MBT; BioTru Ceramic Bracket .018 MBT; Ceramic Bracket Translux+ .022 MBT; BioTru Ceramic Bracket .022 MBT; Ceramic Bracket Translux+ .018 Edgewise; BioTru Ceramic .018 Edgewise Bracket; Ceramic Bracket Translux+ .022 Edgewise; BioTru Ceramic .022 Edgewise Bracket; Ceramic Bracket Translux+ .018 Pattern I; BioTru Ceramic Bracket .018 Pattern I; Ceramic Bracket Translux+ .022 Standard I; BioTru Ceramic Bracket .022 Pattern I; Ceramic Bracket Translux+ .018 Standard II; BioTru Ceramic Bracket .018 Standard II; Ceramic Bracket Translux+ .022 Standard II; BioTru Ceramic .022 Standard II Bracket; Ceramic Bracket Translux+ .018 Standard III; BioTru Ceramic Bracket .018 Standard III; Ceramic Bracket Translux+ .022 Standard III; BioTru Ceramic Bracket .022 Standard III; Ceramic Bracket Translux+ .018 Ricketts; Bracket BioTru Ceramic .018 Ricketts; Ceramic Bracket Translux+ .022 Ricketts; Bracket BioTru Ceramic .022 Ricketts; Translux+ Ceramic Bracket .018 Damon High Torque; BioTru Ceramic Bracket .018 Damon High Torque; Translux+ Ceramic Bracket .022 Damon High Torque; BioTru Ceramic Bracket .022 Damon High Torque; Ceramic Bracket Translux+ .018 Damon Low Torque; BioTru Ceramic Bracket .018 Damon Low Torque; Ceramic Bracket Translux+ .022 Damon Low Torque; BioTru Ceramic Bracket .022 Damon Low Torque; Translux+ Ceramic Bracket .018 Damon Standard; BioTru Ceramic Bracket .018 Damon Standard; Translux+ Ceramic Bracket .022 Damon Standard;

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BioTru Ceramic Bracket .022 Damon Standard; Ceramic Bracket Translux+ .018 MBT Mod.; BioTru Ceramic Bracket .018 MBT Mod.; Ceramic Bracket Translux+ .022 MBT Mod.; BioTru Ceramic Bracket .022 MBT Mod.; Ceramic Bracket Translux+ .018 Ponce; BioTru Ceramic Bracket .018 Ponce; Ceramic Bracket Translux+ .022 Ponce; BioTru Ceramic Bracket .022 Ponce; Ceramic Bracket Translux+ .018 Roth Mod.; BioTru Ceramic Bracket .018 Roth Mod.; Ceramic Bracket Translux+ .022 Roth Mod.; BioTru Ceramic Bracket .022 Roth Mod.; Ceramic Bracket Translux+ .018 Face Active; BioTru Ceramic .018 Face Active Bracket; Translux+ Ceramic Bracket .022 Face Active; BioTru Ceramic .022 Face Active Bracket; Ceramic Bracket Translux+ .018 Face Hybrid; BioTru Ceramic .018 Face Hybrid Bracket; Ceramic Bracket Translux+ .022 Face Hybrid; BioTru Ceramic .022 Face Hybrid Bracket; Ceramic Bracket Translux+ .018 Simple; BioTru Ceramic Bracket .018 Simple; Ceramic Bracket Translux+ .022 Simple; Bracket BioTru Ceramic .022 Simple; Ceramic Bracket Translux+ .018 PDM; BioTru Ceramic .018 PDM Bracket; Ceramic Bracket Translux+ .022 PDM; BioTru Ceramic .022 PDM Bracket; Ceramic Bracket Translux+ .018 Jorge Rodrigues; BioTru Ceramic Bracket .018 Jorge Rodrigues; Translux+ Ermic Bracke . 022 Jorge Rodrigues; BioTru Ceramic Bracket .022 Jorge Rodrigues.

1.4. Anvisa Notification:10331430018

2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Mota, 14.

CEP: 14140-000 - Cravinhos - SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Arantes Nogueira Neto - CREA: 601037885

www.aditek.com.br

3. Certifying entity:

Not applicable

4. Usage Indication:

Aditek Ceramic Brackets are indicated as auxiliary components in orthodontic treatment. The purpose of using brackets is to serve as a support and fitting point for wires, preformed arches, orthodontic elastics or other auxiliary products used in conventional orthodontic treatments.

The products were developed to be used in the oral cavity environment, directly on the teeth, with the objective of supporting orthodontic treatments in human beings, promoting a better positioning of the teeth within the dental arches, seeking a balanced occlusion.

Brackets are used in conjunction with arches, wires, springs, bands, elastics and other auxiliary products necessary for orthodontic treatment.

They are indicated to treat patients who have malocclusion problems that require alignment and leveling movements of the teeth in the arches.

They are chosen by orthodontic professionals who seek a more aesthetic clinical care.

NOTE:The choice of material to be used is made by the qualified professional, based on their knowledge acquired during their training together with the diagnosis and treatment plan for each patient.

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5. Users:

The application of Ceramic Brackets is restricted to dental professionals. The use by people outside this context may cause harm to the patient.

6. Performance Features:

As the brackets are glued directly to the surface of the teeth, all the force applied to it is transferred directly to the tooth causing its position to be changed and consequently allowing its correction to the desired correct position.

Commonly, an uninterrupted orthodontic treatment takes from 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion and oral and systemic conditions of the patient, and can be extended according to the complexity of the case.

7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are the aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

8. Warnings:

 Aditek products are designed and manufactured for single use only. Its reuse or reprocessing can cause cross-infection, decrease in gluing efficiency and loss of mechanical properties due to natural wear and forces applied to product removal.

 If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

The product must be applied by a dentist who must follow the instructions for use and guidelines specified by the manufacturer.

The product is safe for use in pregnant women. It is known that due to hormonal changes during pregnancy, women are more susceptible to periodontal problems. During orthodontic treatment, difficulty in cleaning can exacerbate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

The procedure for bonding and removing the brackets on the teeth must be performed according to the instructions for use of the manufacturer of the orthodontic adhesive purchased.

The composition of the product does not require specific procedures for its transport and does not present risks to people.

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The product is supplied in a non-sterile condition, however it has proven biocompatibility, requiring no cleaning, disinfection or sterilization measures, it is up to the professional to perform the sterilization.

The professional must use good clinical practices for the disposal of medical products according to current health regulations.

Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

In the event of serious incidents with the product, Aditek do Brasil must be notified to the competent authority at the national level in which the dentists established.

9. Contraindications:

- Patients with poor oral hygiene,
- Patients unable to cooperate with treatment,
- Illness or limitation that may make treatment difficult,
- Existing bone or root resorption,
- Decalcification of existing tooth enamel,
- Existing periodontal complications.

NOTE: Aditek do Brasil S.A. supplies products only to dental professionals. The orthodontist is entirely responsible for identifying and/or possible contraindications that may prevent the use of the product.

10. Residual Risks:

All finished orthodontic treatments are subject to relapse. After treatment, control must be determined by the professional in order to avoid or minimize relapse.

11. Unwanted Side Effects:

The use of the medical device can lead to discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort and pain.

12. Composition:

Non-Toxic Monocrystalline Alumina Ceramic (Sapphire)
Chemical composition limits (%)
100%

Non-Toxic Polycrystalline Alumina Ceramic (Translux)
Chemical composition limits (%)
100%

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 10 years

14. Symbols and Meanings:

	Manufacturer		Do not use if package is damaged or tampered with
	Consult instructions for use		Do not reuse the product
	Heads up		Expiration date
	Product reference code		Lot number

15. Patient Information:

The patient must perform good daily hygiene in order to avoid the installation of bacterial plaque and consequent periodontal diseases and to remove food particles that may settle under the clip. The use of an oral irrigator, type Water Pik, is recommended to help clean teeth and brackets.

It is important that the pregnant patient is referred for professional cleaning at regular intervals, due to the increased susceptibility to periodontal problems during pregnancy.

It is important to be careful when chewing certain types of food, especially hard or crunchy ones, as they can come off or break the brackets. These foods must be cut before eating. Hard foods like popcorn and ice should be avoided.

Do not put objects in your mouth, such as pens and pencils, and be careful not to hit or bite the toothbrush structure when brushing your teeth, which can break or loosen the bracket when subjected to a prying movement.

Smoking or chewing tobacco can stain your brackets.

In the event of a malfunction, seek a dentist.

In case of ingestion or aspiration, seek medical attention.

When participating in sports, always wear mouth guards, as impacts can damage the appliances and injure the oral cavity.

The patient must follow the professional's guidelines regarding containment to avoid relapse.

16. Usage Mode:

The steps below should only be performed by dental professionals.

Brackets must be glued directly onto the surface of the teeth at their base, using orthodontic adhesives.

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- Prepare the surface of the teeth according to the instructions for use of the manufacturer of the orthodontic adhesive purchased;
- Glue the brackets on the teeth according to the instructions for use of the manufacturer of the orthodontic adhesive purchased (We suggest the use of the LightBond adhesive - Aditek);
- Position the selected arch according to the treatment phase to be performed;
- Tie the bow with an elastic or metal ligature.

NOTE:Be careful when placing an elastic ligature, so as not to support the instrument under the bracket wing, exerting a lever movement, which could cause the wing to break.

16.1. Bracket Removal:

- Remove the resin around the bracket with a high-speed resin removal cutter;
- Position the bracket removing instrument in the mesiodistal direction and make a clockwise movement. The bracket must come loose, leaving a small portion of resin on the tooth, which must be removed with a high-speed resin removal cutter or resin removing pliers.

NOTE:If you have difficulty removing the bracket, with a diamond bur, section the bracket horizontally, at the height of the slot, taking care not to touch the dental enamel. After sectioning, remove the parts with a suitable instrument.

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