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1. Product's name:

1.1. Technical Name: Orthodontic bracket

1.2. Commercial name: Aditek Orthodontic Brackets

1.3. Commercial Model: Brackets Octopus L Lingual

1.4. Anvisa Notification: 10331430016

2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Mota, 14.

CEP: 14140-000 - Cravinhos - SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Arantes Nogueira Neto - CREA: 601037885

www.aditek.com.br

3. Certifying entity:

Not applicable

4. Usage Indication:

Orthodontic Brackets are indicated as auxiliary components in orthodontic treatment. The purpose of using brackets is to serve as a point of support and fitting for wires, preformed arches or other auxiliary products used in conventional orthodontic treatments.

Standard brackets must receive individual elastic or metallic ligatures.

The products were developed to be used in the oral cavity environment, directly on the teeth, with the aim of supporting orthodontic treatments in human beings, promoting a better positioning of the teeth within the dental arches, seeking a balanced occlusion.

Brackets are used in conjunction with arches, wires, springs, bands, elastics and other auxiliary products necessary for orthodontic treatment.

They are indicated to treat patients aged from 6 years who have malocclusion problems such as prognathism, retrognathism, deep bite, open bite, crowding, anterior crossbite, posterior crossbite and diastema.

They are chosen by orthodontic professionals who seek clinical care for patients with high aesthetic requirements.

NOTE: The choice of material to be used is made by the qualified professional, based on their knowledge acquired during their training together with the diagnosis and treatment plan for each patient.

5. Users:

The application of orthodontic brackets is restricted to dental professionals. The use by people outside this context may cause harm to the patient.

6. Performance Features:

As the brackets are glued directly to the surface of the teeth, all the force applied to it is transferred directly to the tooth causing its position to be changed and consequently allowing its correction to the desired correct position.

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Commonly, an uninterrupted orthodontic treatment takes from 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion and oral and systemic conditions of the patient, and can be extended according to the complexity of the case.

7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are the aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing. Brackets positioned lingual to the teeth have the greatest advantage of performing orthodontic treatment without seeing the elements of the appliances, such as brackets and archwires.

8. Warnings:

Product containing Chromium and Nickel in its composition, which may cause allergic reactions in patients susceptible to allergy to these substances. In case of reaction, remove the device and advise the patient to seek medical attention.

Aditek products are designed and manufactured for single use only. Its reuse or reprocessing can cause cross-infection, decrease in gluing efficiency and loss of mechanical properties due to natural wear and forces applied to product removal.

If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

The product must be applied by a dentist who must follow the instructions for use and guidelines specified by the manufacturer.

The product is not recommended in the MRI environment, as the product will reflect the magnetic waves generated by the CT scanner, preventing or masking the correct reading of the exam. Therefore, brackets should always be removed before head and neck MRIs.

The product is safe for use in pregnant women. It is known that due to hormonal changes during pregnancy, women are more susceptible to periodontal problems. During orthodontic treatment, difficulty in cleaning can exacerbate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

The procedure for bonding and removing the brackets on the teeth must be performed according to the instructions for use of the manufacturer of the orthodontic adhesive purchased.

The composition of the product does not require specific procedures for its transport and does not present risks to people.

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The product is supplied in a non-sterile condition, however it has proven biocompatibility, requiring no cleaning, disinfection or sterilization measures, it is up to the professional to perform the sterilization.

The professional must use good clinical practices for the disposal of medical products according to current health regulations.

Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

The patient must be warned that, at certain moments of the treatment, spaces may appear between the teeth or possible tooth movements that may not aesthetically please the user of the device. This effect is common in all orthodontic treatment, but it deserves professional attention in patients with lingual braces to avoid it, because without the usual coverage of fixed orthodontic appliances, smile esthetics can be temporarily compromised.

In the event of serious incidents with the product, Aditek do Brasil must be notified to the competent authority at the national level in which the dentists established.

9. Contraindications:

- Patients with poor oral hygiene,
- Patients unable to cooperate with treatment,
- Patients who are allergic to chromium and/or nickel,
- Illness or limitation that may make treatment difficult,
- Existing bone or root resorption,
- Decalcification of existing tooth enamel,
- Existing periodontal complications.

NOTE: Aditek do Brasil S.A. supplies products only to dental professionals. It is the sole responsibility of the dentist identification and/or possible contraindication that may prevent the use of the product.

10. Residual Risks:

All finished orthodontic treatments are subject to relapse. After treatment, control must be determined by the professional in order to avoid or minimize relapse.

11. Unwanted Side Effects:

Device usage doctor It can lead to discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort and pain.

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12. Composition:

| Stainless Steel AISI 17-4 - Bracket | | | | | | | | |
|-------------------------------------|--------|---------|---------|--------|--------------|------------|------------|---------------|
| Chemical composition limits (%) | | | | | | | | |
| Ç | Mn | P | s | yes | Cr | Ni | Ass | Faith |
| ≤ 0.070 | ≤ 1.00 | ≤ 0.040 | ≤ 0.030 | ≤ 1.00 | 15.0 to 17.5 | 3.0 to 5.0 | 3.0 to 5.0 | Remainin g |
| ASTM:A564 | | | | | | | | |

13. Validity:

 10 years.

14. Symbols and Meanings:

| | | | |
|---|---|---|--------------------------------------|
|  | Manufacturer |  | Expiration date |
|  | Consult instructions of use |  | Product contains chromium and nickel |
|  | Heads up |  | Do not reuse the product |
|  | Do not use if package is damaged or tampered with |  | Lot number |
|  | Product reference code | | |

15. Patient Information:

Product containing Chromium and Nickel in its composition, which may cause allergic reactions in people susceptible to these components.

The patient must perform good daily hygiene in order to avoid the installation of bacterial plaque and consequent periodontal diseases.

It is important that the pregnant patient is referred for professional cleaning at regular intervals, due to the increased susceptibility to periodontal problems during pregnancy.

It is important to be careful when chewing certain types of food, especially hard or crunchy ones, as they can come off or break the brackets. These foods must be cut before eating. Hard foods like popcorn and ice should be avoided.

Do not put objects in your mouth, such as pens and pencils, and be careful not to hit or bite the toothbrush structure when brushing your teeth, which can break or loosen the bracket when subjected to a prying movement.

In the event of a malfunction, seek a dentist.

It is important for the patient to know that, as with all fixed appliances, some time must be waited for complete adaptation with the presence of the lingual appliance.

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Provide protective materials (protective wax) until complete adaptation of the patient, which is expected to occur between one and three weeks.

In case of ingestion or aspiration, seek medical attention.

The patient must follow the professional's guidelines regarding containment to avoid relapse.

The product is not recommended in the MRI environment.

16. Usage Mode:

The following steps should only be performed by dental professionals.

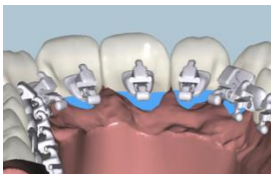
In Lingual Orthodontics, it is recommended that a laboratory process of indirect bonding be followed to:

- Greater assembly accuracy;
- Carry out the modeling of the bases of the brackets;
- Individualize the final dental positioning (individualized prescription).

For this purpose, setups must be built to establish the final positioning of the teeth and to place the lingual brackets on these.

The recommended height for Octopus L brackets is the one that allows accommodation as close as possible to the concavity of the brackets bases in the convexity located in the palatal/lingual cervical third of the anterior teeth.

In this position, a space of 0.5 to 1.0 mm is expected to be established, from the base of the brackets to the gingival margin.

The height of the posterior teeth brackets must follow a horizontal plane defined by the slot of the anterior teeth in a digital or manual setup.

The professional should seek information about the software recommended by the manufacturer for the digital positioning of lingual brackets. More information can be found at www.octopusbraces.com

Services are available (not included in the sale of this product) that include the following guidelines:

- 3- Development of a virtual setup and positioning of the brackets;
- 4- Manufacture of individualized transfer devices for indirect bonding of brackets.

See www.octopusbraces.com for more information.

With the lingual brackets positioned in their respective transfer trays, the surface of the teeth must be prepared according to the instructions for use of the manufacturer of the orthodontic adhesive purchased and proceed to the indirect bonding of the brackets.

Keep the trays in storage for eventual take-offs.

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After treatment, remove the brackets with appropriate pliers and the orthodontic adhesive from the teeth, according to the manufacturer's instructions for use.

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