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1. Product's Name:

1.1. Technical Name: Components for orthodontics

1.2. Commercial Name: Aditek Aesthetic Brackets and Accessories

1.3. Commercial Model: Slide Bracket H5 Clear 018 Roth; Slide Bracket H5 Clear 022 Roth; Slide H5 Clear 018 MBT bracket; Slide H5 Clear 022 MBT bracket; Slide Bracket H5 Clear 018 Damon Super Torque; Slide Bracket H5 Clear 022 Damon Super Torque; Slide Bracket H5 Clear 018 Damon Low Torque; Slide Bracket H5 Clear 022 Damon Low Torque; Slide Bracket H5 Clear 018 Damon Standard; Slide Bracket H5 Clear 022 Damon Standard; Slide Bracket H5 Clear 018 MBT Mod; Slide Bracket H5 Clear 022 MBT Mod; Slide Bracket H5 Clear 018 Roth Mod; Slide Bracket H5 Clear 022 Roth Mod; Slide Bracket H5 Clear 018 Jorge Rodrigues; Slide Bracket H5 Clear 022 Jorge Rodrigues; Bracket BioCrystal 018 Roth; Bracket BioCrystal 022 Roth; Bracket BioCrystal 018 MBT; Bracket BioCrystal 022 MBT; EasyClip Crystal 018 Roth Bracket; EasyClip Crystal 022 Roth Bracket; EasyClip Crystal Bracket 018 MBT; EasyClip Crystal Bracket 022 MBT; EasyClip Crystal Bracket 022 Damon Super Torque; EasyClip Crystal 022 Damon Low Torque Bracket; EasyClip Crystal 022 Damon Standard Bracket; Round EasyClear Aesthetic Button; Oval EasyClear Aesthetic Button; EasyClear Trapezoidal Aesthetic Button.

1.4. Anvisa Registration: 10331439003

2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Mota, 14.

CEP: 14140-000 - Cravinhos - SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Arantes Nogueira Neto - CREA: 601037885

www.aditek.com.br

3. Certifying entity:

Not applicable.

4. Usage Indication:

Aditek Esthetic Brackets and Accessories are indicated as aids in orthodontic treatment. The purpose of using Aditek esthetic brackets and accessories is to serve as a support and fitting point for wires, preformed arches, orthodontic elastics or other auxiliary products used in orthodontic treatments. The products were developed to be used in the oral cavity environment, directly on the teeth, with the aim of supporting orthodontic treatments in human beings, promoting a better positioning of the teeth within the dental arches, seeking a balanced occlusion.

They are indicated to treat patients who have malocclusion problems that require alignment and leveling movements of the teeth in the arches.

They are chosen by orthodontic professionals who seek a more aesthetic clinical care.

Brackets are used in conjunction with arches, wires, springs, bands, elastics and other auxiliary products necessary for orthodontic treatment.

NOTE: Aditek's Aesthetic Brackets were designed to be used with light strength, thermal memory archwires, with a maximum caliber of up to .017”x.025”, according to the recommendations presented in the table below:

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Arc Sequence Recommendation		
Phase I	Alignment Phase	arc deflection
<ul style="list-style-type: none"> • Alignment and leveling • arcade development 	.013” Contour NiTi .014” Contour NiTi or Heat Activated NiTi	More than 0.5mm
Phase II	Leveling Phase	
<ul style="list-style-type: none"> • complete rotation • torque control • angulation correction • Flatten Curve of Spee 	.016” Contour NiTi or Thermoactivated NiTi .014”x.025” Contour NiTi .016”x.025” Contour NiTi .016”x.022” Nitinol Reverse Curve	0.35mm to 0.5mm
Phase III	Work Phase	
<ul style="list-style-type: none"> • Closing spaces • Anterior retraction with torque control 	.016”x.022” Thermoactivated NiTi or .016”x.025” Contour NiTi	Less than 0.35mm
Phase IV	Finishing Phase	
<ul style="list-style-type: none"> • midline correction • Root alignment and final torque control 	.016”x.022” TMA .017”x.025” Thermoactivated NiTi	Less than 0.15mm

NOTE: The choice of material to be used is made by the qualified professional, based on their knowledge acquired during their training together with the diagnosis and treatment plan for each patient.

5. Users:

The application of Aditek Esthetic Brackets and Accessories is restricted to dental professionals. The use by people outside this context may cause harm to the patient.

6. Performance Features:

As Aditek Brackets and Accessories are glued directly to the surface of the teeth, all the force applied to it is transferred directly to the tooth causing its position to be changed and consequently allowing its correction to the desired correct position.

Commonly, an uninterrupted orthodontic treatment takes from 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion and oral and systemic conditions of the patient, and can be extended according to the complexity of the case.

7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are the aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

8. Warnings:

 Aditek products are designed and manufactured for single use only. Its reuse or reprocessing can cause cross-infection, decrease in gluing efficiency and loss of mechanical properties due to natural wear and forces applied to product removal.

 If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

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Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

The product must be applied by a dentist who must follow the instructions for use and guidelines specified by the manufacturer.

The product is safe for use in pregnant women. It is known that due to hormonal changes during pregnancy, women are more susceptible to periodontal problems. During orthodontic treatment, difficulty in cleaning can exacerbate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

The procedure for bonding and removing brackets and accessories on the teeth must be performed according to the instructions for use of the manufacturer of the orthodontic adhesive purchased.

The composition of the product does not require specific procedures for its transport and does not present risks to people.

The product is supplied in a non-sterile condition, however it has proven biocompatibility. If cleaning or disinfection is necessary, dip the product in 70% alcohol, it is up to the professional to perform the sterilization.

The professional must use good clinical practices for the disposal of medical products according to current health regulations.

Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

In the event of serious incidents with the product, Aditek do Brasil must be notified to the competent authority at the national level in which the dentist is established.

9. Contraindications:

- Patients with poor oral hygiene;
- Patients with inability to cooperate with treatment;
- Illness or limitation that may make treatment difficult;
- Existing bone or root resorption;
- Decalcification of existing tooth enamel;
- Existing periodontal complications.

NOTE: Aditek do Brasil S.A. supplies products only to dental professionals. It is the sole responsibility of the dentist the identification and/or possible contraindications that may prevent the use of the product.

10. Residual Risks

All finished orthodontic treatments are subject to relapse. After treatment, control must be determined by the professional in order to avoid or minimize relapse.

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11. Unwanted Side Effects

The use of the medical device may cause discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort and pain.

12. Composition:

Polycarbonate
Chemical composition limits (%)
100%

13. Validity:

 7 years.

14. Symbols and Meanings:

	Manufacturer		Product reference code
	Consult instructions for use		Expiration date
	Heads up		Do not reuse the product
	Do not use if package is damaged or tampered with		Lot number

15. Patient Information:

The patient must perform good daily hygiene in order to avoid the installation of bacterial plaque and consequent periodontal diseases and to remove food particles that may settle in the brackets or accessories. The use of an oral irrigator, type Water Pik, is recommended to help clean teeth and brackets.

It is important that the pregnant patient is referred for professional cleaning at regular intervals, due to the increased susceptibility to periodontal problems during pregnancy.

It is important to be careful when chewing certain types of food, especially hard or crunchy ones, as they can come off or break the brackets. These foods must be cut before eating. Hard foods like popcorn and ice should be avoided.

Do not put objects in your mouth, such as pens and pencils, and be careful not to hit or bite the toothbrush structure when brushing your teeth, which can break or loosen the bracket when subjected to a prying movement.

Smoking or chewing tobacco can stain your brackets.

In the event of a malfunction, consult a dentist.

In case of ingestion or aspiration, seek medical attention.

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When participating in sports, always wear mouth guards, as impacts can damage the appliances and injure the oral cavity.

The patient must follow the professional's guidelines regarding containment to avoid relapse.

16. Usage Mode:

The steps below should only be performed by dental professionals.

Aditek Esthetic Brackets and Accessories must be glued directly onto the surface of the teeth by their base, using orthodontic adhesives, following the adhesive manufacturer's instructions for use.

- Prepare the surface of the teeth according to the instructions for use of the manufacturer of the orthodontic adhesive purchased;
- Glue the brackets on the teeth according to the instructions for use of the manufacturer of the orthodontic adhesive purchased (We suggest the use of the Light Bond adhesive - Aditek);
- Position the selected arch according to the treatment phase to be performed;
- Tie the bow with an elastic or metal ligature.

NOTE: Be careful when placing an elastic ligature, so as not to support the instrument under the bracket wing, exerting a lever movement, which could cause the wing to break.

16.1. Bracket and/or Accessory Removal:

- Remove the resin around the bracket with a high-speed resin removal cutter;
- Position the bracket removing instrument in the mesiodistal direction and make a clockwise movement;
- The bracket must come loose, leaving a small portion of resin on the tooth, which must be removed with a high-speed cutter for resin removal or resin removing pliers;

NOTE: If you have difficulty removing the bracket and/or accessory, with a diamond bur, section the bracket and/or accessory horizontally, at the height of the slot, taking care not to touch the dental enamel. After sectioning, remove the parts with a suitable instrument.

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