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## 1. Product's Name:

**1.1. Technical Name:** Instruments for Dental Use

**1.2. Commercial Name:** Aditek Non-Articulating Non-Cutting Dental Instruments

**1.3. Commercial Model:** Explorer D; Explorer M; Explorer S; Tweezers C; Tweezers P; Tweezers U.

**1.4. Anvisa Notification:** 10331430021

## 2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Motta, 14.

CEP: 14140-000 - Cravinhos - SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Arantes Nogueira Neto - CREA: 601037885

[www.aditek.com.br](http://www.aditek.com.br)

## 3. Certifying entity:

SIQ Ljubljana

SLO-1000 Ljubljana, Trzaska basket 2

tel.: +386 1 4778 100

[info@siq.si](mailto:info@siq.si)

[www.siq.si](http://www.siq.si)

## 4. Usage Indication:

Dental instruments are designed for use by healthcare professionals experienced in specialized and appropriate procedures. It is the responsibility of the practitioner to familiarize himself with the proper techniques.

Explorers are used in orthodontic procedures to open the self-ligating bracket clip. Tweezers are used in orthodontic procedures to open and close the self-ligating bracket clip.

## 5. Users:

The use of Aditek non-articulated non-cutting dental instruments is restricted to dental professionals, as their functions and attributions are maintained by correct handling and especially the correct means of cleaning after use. The use by people outside this context may cause harm to the patient.

## 6. Performance Features:

Aditek non-articulating non-cutting dental instruments are used by dental professionals in orthodontic procedures for opening and closing the self-ligating bracket clip, aiming to improve the most diverse procedures performed in the dental office.

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## 7. Clinical Benefits:

The clinical benefits that these systems bring to the patient's life is to maintain good instruments so that dentists can take care of the oral health of their patients.

## 8. Warnings:

Product containing Chromium and Nickel in its composition, which may cause allergic reactions in patients susceptible to allergy to these substances. In case of reaction, advise the patient to seek medical attention.

If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

Product must be sterilized in an autoclave, following the product manufacturer's recommendations. Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

Store at room temperature away from moisture, direct heat and infectious agents. The product after sterilization must be stored in non-violable packaging, always check the sterilization expiration date, as well as the integrity of the packaging.

Check, after each use, that the instruments have not suffered any damage.

Always use the appropriate instruments for each type of procedure, do not substitute an instrument for another that is not appropriate for the intended use.

The composition of the product does not require specific procedures for its transport and does not present risks to people.

The product is supplied in a non-sterile condition, therefore it must be sterilized before use, according to the sterilization techniques of surgical instruments. After use, the product must be washed, re-sterilized and dried immediately.

Never store clean instruments in boxes that are stained or with marked scratches (they may be sources of contamination).

Instant sterilization procedures should not be used.

Do not use radiation sterilization, formaldehyde sterilization, ethylene oxide sterilization, dry heat sterilization or plasma sterilization.

The professional must use good clinical practices for the disposal of medical products according to current health regulations.

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Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

In case of serious incidents with the product, Aditek do Brasil must be notified to the competent authority in the national scope where the dentist is established.

### 9. Contraindications:

The orthodontist is entirely responsible for identifying and/or possible contraindications that may prevent the use of the product.

### 10. Residual Risks:

Not applicable.

### 11. Unwanted Side Effects:

The incorrect use of dental instruments can cause injuries and abrasions in the hands of dentists, and also, dental fragments or liquids can be projected with consequent risk of skin or eye injury.

### 12. Composition:

Explorer Tip - Stainless Steel AISI 17-4								
Chemical composition limits (%)								
C	Mn	P	s	yes	Cr	Ni	Ass	Faith
≤ 0.070	≤ 1.00	≤ 0.040	≤ 0.030	≤ 1.00	15.0 to 17.5	3.0 to 5.0	3.0 to 5.0	Remaining
ASTM:A564								

### 13. Validity:

 undetermined.

### 14. Symbols and Meanings:

	Manufacturer		Product reference code
	Consult instructions for use		Expiration date
	Heads up		Product contains chromium and nickel
	Do not use if package is damaged or tampered with		Lot number

### 15. User Information:

Dental instruments do not pose a danger to their users and do not have restrictions of use as long as their limitations of use and their original purpose are observed, that is, the use and application of the

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instrument in situations not indicated for the instrument are not the responsibility of the manufacturer. but from consumer misuse.

The Product is a non-sterile material for permanent use, therefore it must be sterilized before and after its use, according to surgical instrument sterilization techniques. After use, the product must be washed and dried immediately. After sterilization, the product must be stored in non-violable packaging, following basic storage precautions.

The heating and cooling of the instruments cause dilations and contractions. Unregulated autoclaves may create residual moisture which may stain and/or corrode surgical instruments.

Always check the sterilization expiration date, as well as the integrity of the packaging.

Product containing chromium and nickel in its composition, which may cause allergic reactions in people susceptible to these components.

## 16. Usage Mode:

Product supplied non-sterile.

**Important:** Keep the instrument clean before using it to ensure uniform contact and forces.

For correct and safe use, the following procedures must be adopted:

- 1) Select the corresponding instruments so that they are compatible with the procedures to be performed;
- 2) In each case the instruments must be checked to ensure their proper use. For each purpose, the appropriate model is used;
- 3) Instruments must be autoclaved before use. After use, they should be washed with enzymatic products or with water and neutral detergent (we recommend the use of one with neutral pH - 7.0, easy to rinse, without chlorine, without sequestering ingredients for hard water and that has low sparkling wine) and can be resterilized;
- 4) Open the surgical package aseptically at the time of use.

### 16.1. Bracket Opening:

Place the instrument in the hole in the clip, located below the slot, between the lower fins and perpendicular to the buccal face of the tooth. Turn the instrument gently to the right or left; if the piece has a hook, the rotation must be in the opposite direction.

### 16.2. Steps to Reprocess Orthodontic Instruments:

#### 16.2.1. Manual Cleaning:

Remove coarse impurities with a soft bristle brush. NEVER use metal or steel wool brushes.

Automatic cleaning using the automatic washing unit

Place instruments in cassettes or other suitable tray systems compatible with the washing unit, according to the manufacturer's instructions.

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### 16.2.2. Sterilization:

Follow all instructions provided by the sterilizer equipment manufacturer.

Minimum Times for Gravity Displacement Steam Sterilization Cycles		
Item	Exposure time at 121° C (250° F)	Drying Time
	30 minutes	At least 30 minutes
Minimum Times for Steam Sterilization Cycles with Dynamic Air Removal		
Item	Exposure time at 132°C (270°F)	drying time
	4 minutes	At least 30 minutes

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