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Not applicable.

4. Usage indication:

The VisProtek face protection mask is indicated for the protection of the user's face against impacts from objects and flying particles. It has a simple structure for size adjustment. The use of a protective mask does not exempt the use of other protective equipment (such as an N95 mask, surgical mask and goggles).

5. Users:

Use indicated for use by the general population in all environments and health professionals working in hospitals, clinics, health units, dental offices or any activity that involves public service.

6. Performance Features:

The VisProtek Face Mask is a transparent physical barrier of protection that prevents saliva or fluids when talking, coughing or sneezing from coming into contact with the wearer.

7. Clinical Benefits:

The main benefit of this product is to offer greater facial protection to its user, less chances of contamination of surgical masks and contamination by the eyes, through a physical barrier.

8. Warnings:

 If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

 Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

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- The composition of the product does not require specific procedures for its transport and does not present risks to people;
- The product is supplied in a non-sterile condition and can be reused as long as it is sanitized before and after use;
- The professional must use good clinical practices for the disposal of medical products according to current health regulations;
- Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use;
- If serious incidents occur with the product, Aditek do Brasil and the competent authority at the national level in which the product is established must be reported.

9. Contraindications:

This product is not intended for fire fighting and arc flash protection. Do not use face shields to protect against infrared radiation. This equipment is not suitable for work where protection against heat radiation is required, such as furnaces, casting, heat treatment, etc. Do not use for risks arising from plasma and/or laser welding, cutting and welding processes.

10. Residual Risks:

Not applicable.

11. Unwanted Side Effects:

Not applicable.

12. Composition:

| | |
|--|--------------------|
| face mask holder | |
| Chemical composition limits (%) | |
| High impact polystyrene | |
| 100% | |
| visor lens | |
| Chemical composition limits (%) | |
| Polyethylene Ethylene Glycol Terephthalate (PETG) | |
| 100% | |
| Adjustment elastic | |
| Chemical composition limits (%) | |
| Polyester | elastodiene |
| 59% | 41% |
| forehead support | |
| Chemical composition limits (%) | |
| EVA (Ethyl Vinyl Acetate) Adhesive Tape | |
| 100% | |

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 2 years

14. Symbols and Meanings:

| | | | |
|---|---|---|-------------------------|
|  | Manufacturer |  | Product reference code |
|  | Consult instructions for use |  | Expiration date |
|  | Heads up |  | Keep away from sunlight |
|  | Do not use if package is damaged or tampered with |  | Lot number |

15. User Information:

Sanitize before and after use. It is recommended that you clean the face shield with room temperature water and soap before disinfecting it.

The product can be easily sanitized with 70% alcohol or Sodium Hypochlorite.

Do not expose the Easy Protector to extreme temperatures. Do not use heat sources to sterilize the face shield.

Do not use abrasive products to clean and/or disinfect the face shield, never use solvents and other chemical products to remove stains or solid residues.
Make sure the face shield is correctly fitted to the face.

Always inspect your face shield before use to ensure it performs well and is safe.

Immediately replace the product when it has holes, scratches or any other damage.

In case of discomfort, discontinue use.

16. Usage mode:

The VisProtek face shield must fit tightly to the wearer's forehead through the elastic openings, but must not exert excessive force to the point of discomfort.

- Equipment for individual use. Use must be personal and non-transferable;
- Use in conjunction with other personal protective equipment, such as an N95 mask or surgical mask and goggles;
- Sanitize before first use and after contact with patients;
- Wash with water and soap/detergent and only after this cleaning, carry out the disinfection process;

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- Disinfect with 70% alcohol or sodium hypochlorite and in accordance with the Hospital Infection Commission;
- It should not be exposed to extreme temperatures;
- Avoid prolonged use of this equipment. In case of discomfort, discontinue use;
- This product may have a short shelf life. Do not bend the lens or fitting cuts and handle with care to preserve its lifespan;
- Dispose of as medical waste.

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