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1. Product Name:

1.1. Technical name: Dental Whitening Agents

1.2. Commercial name: Whitening Gel

1.3. Commercial Model: Replacements: 1 - 01 syringe 3 ml of 10% gel; 2- 01 syringe 5 ml of 10% gel; 3 - 01 syringe 10 ml of 10% gel; 4 - 01 syringe 3 ml of 16% gel; 5 - 01 syringe 5 ml of 16% gel; 6 - 01 syringe 10 ml of 16% gel; 7 - 01 syringe 3 ml of 22% gel; 8 - 01 syringe 5 ml of 22% gel; 9 - 01 syringe 10 ml of 22% gel; 10 - 01 syringe 3 ml of 35% gel; 11 - 01 syringe 5 ml of 35% gel; 12 - 01 syringe 10 ml of 35% gel; 13 - 01 syringe 3 ml of 44% gel; 14 - 01 syringe 5 ml of 44% gel; 15 - 01 syringe 10 ml of 44% gel; 16 - 01 pen 2 ml of 10% gel; 17 - 01 pen 4 ml of 10% gel; 18 - 01 pen 2 ml of 16% gel; 19 - 01 pen 4 ml of 16% gel; 20 - 01 pen 2 ml of 22% gel; 21 - 01 pen 4 ml of 22% gel; Mini Kits: 22 – Aditek Mini Kit 10% - 9 ml - 3 syringes of whitening gel 10% containing 3 ml each + 1 color scale; 23 – Aditek Mini Kit 16% - 9 ml - 3 syringes of whitening gel 16% containing 3 ml each + 1 color scale; 24 - Aditek Whitening Gel Mini Kit 22% - 9 ml - 3 syringes of whitening gel 22% containing 3 ml each + 1 color scale; 25 - Aditek Whitening Gel Mini Kit 35% - 9 ml - 3 syringes of whitening gel 35% containing 3 ml each + 1 color scale; 26 - Aditek Whitening Gel Mini Kit 44% - 9 ml - 3 syringes of whitening gel 44% containing 3 ml each + 1 color scale; 27 - Aditek Whitening Gel Mini Kit 10% - 15 ml - 3 syringes of whitening gel 10% containing 5 ml each + 1 color scale; 28 - Aditek Whitening Gel Mini Kit 16% - 15 ml - 3 syringes of whitening gel 16% containing 5 ml each - 3 syringes of whitening gel 22% containing 5 ml each + 1 color scale; 30 - Aditek Whitening Gel Mini Kit 35% - 15 ml - 3 syringes of whitening gel 35% containing 5 ml each + 1 color scale; 31 - Aditek Whitening Gel Mini Kit 44% - 15 ml - 3 syringes of whitening gel 44% containing 5 ml each + 1 color scale; 32 - Aditek Whitening Gel Mini Kit 10% - Pen 6 ml - 3 whitening gel pens 10% containing 2 ml each + 1 color scale; 33 - Aditek. Whitening. Gel Mini Kit 16% - Pen 6 ml - 3 whitening gel pens 16% containing 2 ml each + 1 color scale; 34 - Aditek Whitening Gel Mini Kit 22% - Pen 6 ml - 3 whitening gel pens 22% containing 2 ml each + 1 color scale; 35 - Aditek Whitening Gel Mini Kit 10% - Pen 8 ml - 2 whitening gel pens 10% containing 4 ml each + 1 color scale; 36 - Aditek Whitening Gel Mini Kit 16% - Pen 8 ml - 2 whitening gel pens 16% containing 4 ml each + 1 color scale; 37 - Aditek Whitening Gel Mini Kit 22% - Pen 8 ml - 2 whitening gel pens 22% containing 4 ml each + 1 color scale; 38 - Aditek Whitening Gel Mini Kit 10% - Pen 12 ml - 2 whitening gel pens 10% containing 4 ml each + 1 color scale; 39 - Aditek Whitening Gel Mini Kit 16% - Pen 12 ml - 3 whitening gel pens 16% containing 4 ml each + 1 color scale; 40 - Aditek Whitening Gel Mini Kit 22% - Pen 12 ml - 3 whitening gel pens 22% containing 4 ml each + 1 color scale; KITS: 41 - Aditek Whitening Gel Kit 35% - 6 ml - 3 syringes of whitening gel 35% containing 2 ml each + 1 syringe of gingival barrier with 3 ml + 1 syringe of desensitizer with 2 ml + 1 color scale; 42 - Aditek Whitening Gel Kit 44% - 6 ml - 3 syringes of whitening gel 44% containing 2 ml each. + 1 syringe of gingival barrier with 3 ml + 1 syringe of desensitizer with 2 ml + 1 color scale; 43 - Aditek Whitening Gel Kit 10% - Pen 6 ml - 3 whitening gel pens 10% containing 2 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 44 - Aditek Whitening Gel Kit 16% - Pen 6 ml - 3 whitening gel pens 16% containing 2 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 45 - Aditek Whitening Gel Kit 22% - Pen 6 ml - 3 whitening gel pens 22% containing 2 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 46 - Aditek Whitening Gel Kit 10% - Pen 8 ml - 2 whitening gel pens 10% containing 4 ml each + 1 tray with led light + 3 smart phone adapters + 1 color scale; 47 - Aditek Whitening Gel Kit 16% - Pen 8 ml - 2 whitening gel pens 16% containing 4 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 48 - Aditek

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Whitening Gel Kit 22% - Pen 8 ml - 2 whitening gel pens 22% containing 4 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 49 - Aditek Whitening Gel Kit 10% - Pen 12 ml - 3 whitening gel pens 10% containing 4 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 50 - Aditek Whitening Gel Kit 16% - Pen 12 ml - 3 whitening gel pens 16% containing 4 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 51 - Aditek Whitening Gel Kit 22% - Pen 12 ml - 3 whitening gel pens 22% containing 4 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; Mini Kits: 52 - Orthopride Whitening Gel Mini Kit 10% - 9 ml - 3 syringes of whitening gel 10% containing 3 ml each + 1 shade scale; 53 - Orthopride Whitening Gel Mini Kit 16% - 9 ml - 3 syringes of whitening gel 16% containing 3 ml each + 1 color scale; 54. - Orthopride Whitening Gel Mini Kit 22% - 9 ml - 3 syringes of whitening gel 22% containing 3 ml each + 1 color scale; 55 - Orthopride Whitening Gel Mini Kit 35% - 9 ml - 3 syringes of whitening gel 35% containing 3 ml each + 1 color scale; 56 - Orthopride Whitening Gel Mini Kit 44% - 9 ml - 3 syringes of whitening gel 44% containing 3 ml each + 1 color scale; 57 - Orthopride Whitening Gel Mini Kit 10% - 15 ml - 3 syringes of whitening gel 10% containing 5 ml each + 1 color scale; 58 - Orthopride Whitening Gel Mini Kit 16% - 15 ml - 3 syringes of whitening gel 16% containing 5 ml each + 1 color scale; 59 - Orthopride Whitening Gel Mini Kit 22% - 15 ml - 3 syringes of whitening gel 22% containing 5 ml each + 1 color scale; 60 - Orthopride Whitening Gel Mini Kit 35% - 15 ml Mini Kit (3 syringes of 5 ml) - the mini kit is presented in the Carbamide Peroxide concentration of - 3 whitening gel syringes 35% containing 5 ml each + 1 scale by heart; 61 - Orthopride Whitening Gel Mini Kit 44% - 15 ml - 3 syringes of whitening gel 44% containing 5 ml each + 1 color scale; 62 - Orthopride Whitening Gel Mini Kit 10% - Pen 6 ml - 3 whitening gel pens 10% containing 2 ml each + 1 color scale; 63 - Orthopride Whitening Gel Mini Kit 16% - Pen 6 ml - 3 whitening gel pens 16% containing 2 ml each + 1 color scale; 64 - Orthopride Whitening Gel Mini Kit 22% - Pen 6 ml - 3 whitening gel pens 22% containing 2 ml each + 1 color scale; 65 - Orthopride Whitening Gel Mini Kit 10% - Pen 8 ml - 2 whitening gel pens 10% containing 4 ml each + 1 color scale; 66 - Orthopride Whitening Gel Mini Kit 16% - Pen 8 ml - 2 whitening gel pens 16% containing 4 ml each + 1 color scale; 67 - Orthopride Whitening Gel Mini Kit 22% - Pen 8 ml - 2 whitening gel pens 22% containing 4 ml each + 1 color scale; 68 - Orthopride Whitening Gel Mini Kit 10% - Pen 12 ml - 3 whitening gel pens 10% containing 4 ml each + 1. Color scale; 69 - Orthopride Whitening Gel Mini Kit 16% - Pen 12 ml - 3 whitening gel pens 16% containing 4 ml each + 1 color scale; 70 - Orthopride Whitening Gel Mini Kit 22% - Pen 12 ml - 3 whitening gel pens 22% containing 4 ml each + 1 color scale; KITS: 71 - Orthopride Whitening Gel Kit 35% - 6 ml - 3 syringes of whitening gel 35% containing 2 ml each + 1 syringe of gingival barrier with 3 ml + 1 syringe of desensitizer with 2 ml + 1 color scale; 72 - Orthopride Whitening Gel Kit 44% - 6 ml - 3 syringes of whitening gel 44% containing 2 ml each + 1 syringe of gingival barrier with 3 ml + 1 syringe of desensitizer with 2 ml + 1 shade scale; 73 - Orthopride Whitening Gel Kit 10% - Pen 6 ml - 3 whitening gel pens 10% containing 2 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 74 - Orthopride Whitening Gel Kit 16% - Pen 6 ml - 3 whitening gel pens 16% containing 2 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 75 - Orthopride Whitening Gel Kit 22% - Pen 6 ml - 3 whitening gel pens 22% containing 2 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 76 - Orthopride Whitening Gel Kit 10% - Pen 8 ml - 2 whitening gel pens 10% containing 4 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 77 - Orthopride Whitening Gel Kit 16% - Pen 8 ml - 2 whitening gel pens 16% containing 4 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 78 - Orthopride Whitening Gel Kit 22% - Pen 8 ml - 2 whitening gel pens 22% containing 4 ml each + 1

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Tray with LED light + 3 adapters for smart phone + 1 color scale; 79 - Orthopride Whitening Gel Kit 10% - Pen 12 ml - 3 whitening gel pens 10% containing 4 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 80 - Orthopride Whitening Gel Kit 16% - Pen 12 ml - 3 whitening gel pens 16% containing 4 ml each + 1 Tray with LED light + 3 adapters for smart phone + 1 color scale; 81 - Orthopride Whitening Gel Kit 22% - Pen 12 ml - 3 whitening gel pens 22% containing 4 ml each + 1 tray with LED light + 3 smart phone adapters + 1 color scale.

1.4. Anvisa Notification: 10331430035

2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Mota, 14.

CEP: 14140-000 - Cravinhos – SP - Brasil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Manager: José Arantes Nogueira Neto - CREA: 601037885

www.aditek.com.br

3. Certifying Body:

Not applicable.

4. Indication of Use

Whitening Gel is indicated for whitening vital and non-vital teeth, under professional guidance. They are indicated for the following cases:

Vital teeth: naturally dark or yellowish; darkened by age – physiological darkening; darkened or stained due to the incorporation of substances that contain pigments such as coffee, tobacco, tea; with spots due to fluorosis; stained by tetracycline.

Non-vital teeth: in association with internal tooth whitening.

The plastic tray is used to place the gel and position it on the teeth.

The led light to attach to the tray is used to enhance the action of the whitening gel.

Desensitizer is indicated to decrease the sensitivity that can be caused by tooth whitening.

Gingival barrier is indicated as a substitute for the rubber sheet for the protection of the gingival tissue in cases of risk of irritation due to contact with the product used in tooth whitening.

Note: Sale By Prescription.

5. Users:

The use of Whitening Gel is indicated for home use supervised by a professional.

6. Performance Characteristics:

The use of Whitening Gel is aimed at whitening vital and non-vital teeth.

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7. Clinical Benefits:

The main clinical benefit that this product offers to the patient is an aesthetic improvement of the teeth through tooth whitening.

8. Warnings:**Sale Under Prescription.**

Aditek products are designed and manufactured for single use only. Prohibited reprocessing product.

If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

Store the product in a cool, dry place and away from sunlight, keeping it in its own packaging until use.

The product must be applied under the supervision of a trained professional who must follow the instructions for use and guidelines specified by the manufacturer.

The composition of the product does not require specific procedures for its transport and does not pose a risk to people.

Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

If serious incidents occur with the product, Aditek do Brasil must be notified to the competent authority in the national scope where the dentist is established.

9. Contraindications:

- Do not use in patients with recent oral surgery, oral tissues in poor condition, even in the presence of oral wounds or advanced periodontal disease;
- Do not use with inflamed or scarred oral tissues, leaking restorations or decayed teeth;
- Do not use in patients with hypersensitivity to the product;
- Do not use in pregnant or lactating patients and under 15 years of age;
- Do not use in smokers or patients with high alcohol consumption habits;
- Do not ingest acidic foods during treatment, such as fruit juices, soft drinks and coffee, which may favor sensitivity;
- Do not eat foods that contain strong dyes such as coffee, chocolates, açaí, wines or soft drinks, given the possibility of nullifying or reducing the effect of the bleach;
- Occasionally, patients may experience temporary discomfort in their teeth and gums. If the discomfort persists, look for a professional and/or suspend the use of the product.

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NOTE: It is the dentist's sole responsibility to identify and/or possible contraindications that may prevent the use of the product.

10. Residual Risks:

Every finished treatment is subject to relapse. After treatment, control must be determined by the professional in order to avoid or minimize recurrence.

11. Undesirable Side Effects:

Adverse effects are rare, but can include:

- Irritation of oral mucous membranes due to leakage of gel from the tray or mechanical irritation caused by poor adaptation of the tray, which must have smooth and well-finished edges;
- During bleaching, the patient may feel some kind of dental or gingival sensitivity. When in doubt, consult a professional and/or suspend the use of the product.

12. Composition:

	Whitening Gel 10%	Whitening Gel 16%	Whitening Gel 22%	Whitening Gel 35%	Whitening Gel 44%
Ingredients	Chemical Composition Limits (%)	Chemical Composition Limits (%)	Chemical Composition Limits (%)	Chemical Composition Limits (%)	Chemical Composition Limits (%)
Carbamide Peroxide	10%	16%	22%	35%	44%
Glycerin	31%	33%	31%	20%	20%
Water	29,5%	29,5%	25,5%	22,8%	13,8%
Propylene glycol	23%	15%	15%	15%	15%
Xanthan Gum	2,5%	2,5%	2,5%	2,5%	2,5%
Sodium citrate	1%	1%	1%	1%	1,5%
Menthol	1%	1%	1%	1%	1%
Sodium Phytate	0,6%	0,6%	0,6%	0,8%	0,8%
locust bean gum	0,5%	0,5%	0,5%	0,8%	0,5%
Salicylic acid	0,5%	0,5%	0,5%	0,5%	0,5%
Sodium Silicate	0,2%	0,2%	0,2%	0,2%	0,2%
Xylitol	0,1%	0,1%	0,1%	0,1%	0,1%
Tetrasodium pyrophosphate	0,1%	0,1%	0,1%	0,1%	0,1%

Desensitizing Gel	
Ingredients	Chemical Composition Limits (%)
Potassium nitrate	3%
Glycerol	30%
Propylene Glycol	32%
Deionized water	30%
Triethanolamine	1,5%
Carbomer	3%
Carboxymethyl	2%
Poly Vinylpyrrolidone	1%
Menthol	0,4%

Gingival Barrier	
Ingredients	Chemical Composition Limits (%)
Diurethane Dimethacrylate	58%
Silicon dioxide	20%
4-Methacryloxyethyl Trimethyl Anhydrous	12%
Titanium dioxide	5%
Light Initiators	3%
Sodium saccharin	1%
Pigments	1%

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 1 year

14. Symbols and Meanings:

	Manufacturer		Product reference code
	Consult instructions for use		Expiration date
	Attention		Do not reuse the product
	Do not use if package is damaged or tampered with.		Lot number
	Temperature limit		Professional Use
	Protect from light		

15. Patient Information:

Although the product is designed to also be used by the patient, it must be used under the supervision of the professional.

If you have severe sensitivity, reduce the whitening use time, space the applications, or if necessary, interrupt the whitening for a few days and consult a professional again.

Protect away from direct light and heat.

Avoid contact with eyes and skin, avoid ingestion and inhalation.

Wash hands before breaks and immediately after handling the product.

Do not use in pregnant, lactating patients and under 15 years old.

Do not eat, drink or smoke while using trays.

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Do not freeze the product.

Do not use the package if it is broken.

16. Mode of Use:

16.1. Professional Instructions for Product Use:

It is recommended that patients receive a dental examination and prophylaxis prior to starting whitening treatment. It is also important to explain proper application techniques to patients to increase effectiveness and comfort during home use.

Make a note of the patient's tooth shade according to the shade scale, for later comparison of the result.

16.2. Anamnesis:

- Examine the patient in search of cavities, enamel fissures, abrasions, erosions, gingival retractions, infiltration of restorations and anything that may bring suspicion of dentin exposure.
- Periodontal changes should also be checked. Do not use in patients with recent gingival surgery.
- Ask the patient about tooth sensitivity. If so, proceed with the previous desensitization treatment.
- Investigate the possibility of pregnancy or lactation of the patient. In case of doubt, do not indicate bleaching.
- Instruct the patient about the expected results of the treatment. Inform that only the natural tissues of the teeth will undergo bleaching and that restorations made from artificial materials will remain with their original color. Non-vital teeth will also not respond with the same intensity. Intense pigmentation may require bleaching with higher concentrations, applied only by the professional in the office.
- Do not bleach in the presence of the disturbances mentioned above, as well as all others that you deem important. Only after proper repair of all these alterations will the patient be released for the whitening procedure.

16.3. Instructions for Patient Use of the Product

These instructions must be passed on by the dentist to the patient, so that the treatment is effective and safe.

- 1 - Clean the plastic tray and dry it.
- 2 - Brush your teeth.
- 3 - Keep a wide smile and apply a thin layer of gel on the buccal surface of the teeth.
- 4 - In case of domestic use, connect the led light to the Iphone or Android; if use is in the office, apply the led light.
- 5 - Place the tray in your mouth and bite firmly; press the power button.
- 6 - The daily time of application of the gel, as well as the treatment time, must be determined by the Dental Surgeon. Usually the application time is 25 minutes. Do not use the product without the supervision and guidance of this professional.
- 7 - After the application time, remove the gel from the teeth with cotton.
- 8 - Rinse your mouth with warm water.
- 9 - Dental sensitivity may occur during and/or after treatment. The Dental Surgeon must be informed about what happened as well as all the abnormalities that may occur during the treatment.

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10 - After the application session is over, wash the tray well and dry it before storing it. Store it in a clean, dry place, preferably outdoors, to prevent the proliferation of bacteria and fungi. Take care not to overexpose the syringe or gel pen to strong light or heat, as the product may change.

11 - Avoid contact of the product with the skin and eyes. In case of contact with the skin, wash thoroughly with running water and immediately seek medical help, taking the syringe or the product label.

Do not carry out any procedure other than that recommended by your dentist or described in the instructions for use.

16.4. Instructions for Use for the Use of the Desensitizer:

1 - Clean your teeth.

2 - Apply the desensitizing gel evenly over the teeth and wait a minimum of 10 minutes.

The desensitizer can be applied with the aid of the bleaching tray, applying it as if it were the bleach itself, without the need for light.

3 - After the application time, remove the gel from the teeth with cotton and plenty of water. Prevent the patient from swallowing the desensitizer.

4 - Repeat the application of the desensitizer, following the treatment time. The number of applications should not exceed once a day.

16.5. Instructions for Use for Using the Gingival Barrier:

This product is recommended to be applied by the Dental Surgeon.

1. Dry the gums and teeth corresponding to the area you want to protect.

2. Apply a thin layer of the product (1 - 1.5mm thick) on the gingiva and gingival papillae in such a way that complete coverage is obtained and a good field of protection for the whitening procedure.

3. Cure the gingival barrier using 20 to 30 seconds of light from a curing light. It is recommended to proceed with polymerization soon after application of the gingival barrier to prevent any possible irritation.

4. Check that the gingival barrier covers the entire surface of the gingiva and that it has completely cured. Avoid testing it by forcing or trying to lift it, as this could lead to infiltration of the irritating substance.

5. To remove the gingival barrier, simply detach it forward. It should easily come off completely or in large chunks.

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