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Not applicable.

4. Indication of Use:

The 4D Interface (IF 4D) is indicated to serve as a guide to assist in the indirect bonding of brackets, facilitating the positioning on the patient's teeth, respecting the planning made by the dentist, it is formed by trays manufactured with Thermoplastic Resin, the internal one being flexible and the external rigid, which are produced on a model made from impressions of the upper and lower dental arches of each patient.

For the Exceed RX and Exceed LX models, the dentist molds the patient's arches and records the occlusion in PVS or wax to record the patient's initial bite together with the digitized orthodontic documentation (intra and extraoral photos, panoramic radiographs and telerradiography and treatment plan).

For the Exceed TX and Octopus L models, the dentist molds the patient's arches and records the occlusion in PVS or wax to record the patient's initial bite. the movement of the patient's teeth, in order to virtually obtain the expected treatment results and the choice of bracket prescription to be used.

NOTE: The choice of material to be used is made by the qualified professional, based on his knowledge acquired during his training together with the diagnosis and treatment plan for each patient.

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5. Users:

The use of trays for dentistry (4D Interface) is restricted to dental professionals. The use by people outside this context may result in harm to the patient. The professional must select the product to be used according to the desired tooth movement.

6. Performance Characteristics:

The purpose of Interface 4D (IF 4D) is to serve as a guide for the indirect bonding of brackets to the patient's teeth, for correct positioning of the brackets.

7. Clinical Benefits:

The main benefits that orthodontic treatment brings to the patient's life are the aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

8. Warnings:

Aditek products are designed and manufactured for single use only. Its reuse or reprocessing can cause cross-infection, decrease in bonding efficiency and loss of properties.

If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

Store the product in a cool, dry place and away from sunlight, keeping it in its own packaging until use.

The product must be applied by a dentist who must follow the instructions for use and guidelines specified by the manufacturer.

The product is safe for use in pregnant women. It is known that due to hormonal changes during pregnancy, women are more susceptible to periodontal problems. During orthodontic treatment, the difficulty of cleaning can aggravate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

The composition of the product does not require specific procedures for its transport and does not pose a risk to people.

The professional must use good clinical practices for the disposal of medical products in accordance with current health standards.

Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

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If serious incidents occur with the product, Aditek do Brasil must be notified to the competent authority in the national scope where the dentist is established.

9. Contraindications:

- Patients unable to cooperate with treatment,
- Illness or limitation that may make treatment difficult,
- Existing periodontal complications.

NOTE: Aditek do Brasil S.A. supplies products only to dental professionals. It is the dentist's sole responsibility to identify and/or possible contraindications that may prevent the use of the product.

10. Residual Risks

Not applicable.

11. Undesirable Side Effects

Not applicable.

12. Composition:

Outer Tray	
Product	Chemical Composition Limits (%)
Rigid Thermoplastic Resin	100%

Inner Tray	
Product	Chemical Composition Limits (%)
Flexible Thermoplastic Resin	100%

13. Validity:

Indeterminate

14. Symbols and Meanings:

	Manufacturer		Do not reuse the product
	Consult instructions for use		Lot number
	Attention		Product reference code
	Do not use if package is damaged or tampered with.		Expiration date

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15. Patient Information:

This product should only be used by a professional dentist.

The professional must select the product to be used according to the desired tooth movement.

The use of this product does not pose any risk to the patient.

16. Mode of Use:

The steps below should only be performed by dental professionals licensed in Orthodontics.

- The dentist must prepare the patient's teeth to bond the brackets.
- Apply the photopolymerizable resin to the brackets that are positioned in the trays.
- Position the trays correctly on the patient's teeth.
- Apply Curing Light to each bracket.
- Remove rigid outer tray and discard.
- Remove flexible inner tray and discard.

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