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1. Product's Name:

1.1. Technical Name: Wire for Orthodontics

1.2. Commercial Name: Aditek Archwire Sequence

1.3. Commercial Model: Contour NiTi EasyClip+ Passive Arch Sequence w/10; Contour NiTi EasyClip+ Passive Arch Sequence w/8; Contour NiTi Arch Sequence 35° EasyClip w/8; Contour NiTi EasyClip+ Hybrid Arch Sequence w/10; Contour NiTi Arc Sequence 35° Exceed w/8; Contour NiTi Arch Sequence 35° Aesthetic Slide H5 w/10; PDM EasyClip+ Arch Sequence w/10; Thermoactivated Arch Sequence EasyClip+ Interactive w/8; Active Clear Aesthetic Thermoactivated NiTi Arch Sequence w/8; Contour NiTi Arch Sequence 35° EasyClip w/ 28; Modern Thermoactivated NiTi Arch Sequence Ortho Orthognatic c/8; Modern Ortho Standard Cuper NiTi Arch Sequence w/8; Modern Ortho Class III Open Bite Cuper NiTi Arch Sequence w/8; Modern Ortho Deep Bite Cuper NiTi Arch Sequence w/8; Custom Arch Sequence HV EasyClip+ w/17; Seq. 1 Niti Arch Thermo Self-ligating Hybrid System 022 14/18/21X16 C/6; Seq. 2 Arch Niti Thermo Hybrid Self-ligating System 022 14/18/20X20/21X16 C/8; Seq. 3 Niti Arch Thermo Two-Dimensional Hybrid Self-ligating System 14/18/18X18 C/6; Seq. Thermo Niti Arches/Lingual Steel P 12/16/16x16/18x22 w/10; Seq. Thermo Niti Arches/Lingual Steel M 12/16/16x16/18x22 w/10; Seq. Thermo Niti Arches/Lingual Steel G 12/16/16x16/18x22 w/10.

1.4. Anvisa Notification:10331430029

2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Mota, 14.

CEP: 14140-000 - Cravinhos - SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Arantes Nogueira Neto - CREA: 601037885

www.aditek.com.br

3. Certifying entity:

SIQ Ljubljana

SLO-1000 Ljubljana, Trzaska basket 2

tel.: +386 1 4778 100

info@siq.si

www.siq.si

4. Usage Indication:

Aditek Arch Sequences are indicated for movement of teeth in orthodontic treatment, through mechanical force exerted when placed on supports glued to the teeth.

The purpose of the arches is to exert a continuous force that is transferred to the teeth, promoting their movement in orthodontic treatment. They are applied to brackets and tubes.

NOTE: The choice of material to be used is made by the qualified professional, based on their knowledge acquired during their training together with the diagnosis and treatment plan for each patient.

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5. Users:

The application of Aditek Arches is restricted to dental professionals. The use by people outside this context may cause harm to the patient.

6. Performance Features:

Commonly, an uninterrupted orthodontic treatment takes from 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion and oral and systemic conditions of the patient, and can be extended according to the complexity of the case.

7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are the aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

8. Warnings:

Product containing Chromium and Nickel in its composition, which may cause allergic reactions in patients susceptible to allergy to these substances. In case of reaction, remove the device and advise the patient to seek medical attention.

Aditek products are designed and manufactured for single use only. Its reuse or reprocessing may cause cross-infection.

If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

The product must be applied by a dentist who must follow the instructions for use and guidelines specified by the manufacturer.

The product is not recommended in the MRI environment, as the product will reflect the magnetic waves generated by the CT scanner, preventing or masking the correct reading of the exam, so they must always be removed before head and neck MRIs.

The product is safe for use in pregnant women. It is known that due to hormonal changes during pregnancy, women are more susceptible to periodontal problems. During orthodontic treatment, difficulty in cleaning can exacerbate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

The composition of the product does not require specific procedures for its transport and does not present risks to people.

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The product is supplied in a non-sterilized condition, however it has proven biocompatibility and no cleaning, disinfection or sterilization measures are necessary, being at the discretion of the professional to carry out the sterilization.

The professional must use good clinical practices for the disposal of medical products according to current health regulations.

Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

In case of serious incidents with the product, Aditek do Brasil must be reported to the competent authority at the national level.

9. Contraindications:

- Patients with poor oral hygiene,
- Patients unable to cooperate with treatment,
- Patients who are allergic to chromium and/or nickel,
- Illness or limitation that may make treatment difficult,
- Existing bone or root resorption,
- Decalcification of existing tooth enamel,
- Existing periodontal complications.

NOTE: Aditek do Brasil S.A. supplies products only to dental professionals with qualifications in Orthodontics. The orthodontist is entirely responsible for identifying and/or possible contraindications that may prevent the use of the product.

10. Residual Risks:

All finished orthodontic treatments are subject to relapse. After treatment, control must be determined by the professional in order to avoid or minimize relapse.

11. Unwanted Side Effects:

The use of the medical device can lead to discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort and pain.

12. Composition:

Nickel Titanium NiTi Wire									
Chemical composition limits (%)									
Ni	C	with	Ass	H	Faith	nb	N + O	Cr	You
54.5 to 57.0	≤ 0.050	≤ 0.050	≤ 0.010	≤ 0.005	≤ 0.050	≤ 0.025	≤ 0.050	≤ 0.010	Remaining

304V Stainless Steel Wire - Steel Wire								
Chemical composition limits (%)								
C	yes	Mn	P	s	Ni	Cr	No	Faith
≤ 0.030	≤ 0.75	≤ 2.00	≤ 0.045	≤ 0.030	8.0 to 12.0	18.0 to 20.0	≤ 0.10	Remaining

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Titanium Molybdenum Ti-Beta 3 - TMA									
Chemical composition limits (%)									
Mo	Ç	Zr	Sn	H	Faith	O	No	Other Elements	You
10.0 to 13.0	≤ 0.100	4.50 to 7.50	3.75 to 5.25	≤ 0.020	≤ 0.350	≤ 0.18	≤ 0.050	≤ 0.050	Remaining

NT-HR2 Nickel Titanium Wire – Cuper Niti					
Chemical composition limits (%)					
Ni	Ç	Ass	O	Cr	You
49.0 to 50.0	≤ 0.060	5.0 to 6.0	≤ 0.10	0.30 to 0.60	Remaining

13. Validity:

 10 years

14. Symbols and Meanings:

	Manufacturer		Product contains chromium and nickel
	Consult instructions of use		Do not reuse the product
	Heads up		Lot number
	Do not use if package is damaged or tampered with		Authorized representative in the European Community
	Product reference code		Complies with European directive. XXXX represents the number of the certification body
	Expiration date		

15. Patient Information:

Product containing Chromium and Nickel in its composition, which may cause allergic reactions in people susceptible to these components.

The patient must perform good daily hygiene in order to avoid the installation of bacterial plaque and consequent periodontal diseases.

It is important that the pregnant patient is referred for professional cleaning at regular intervals, due to the increased susceptibility to periodontal problems during pregnancy.

Chewing hard foods can cause appliances to break or fall off;
In the event of a malfunction, consult a dentist.

In case of ingestion or aspiration, seek medical attention.

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When participating in sports, always wear mouth guards, as impacts can damage the appliances and injure the oral cavity.

The patient must follow the professional's guidelines regarding containment to avoid relapse.

The product is not recommended in the MRI environment.

16. Usage Mode:

The steps below should only be performed by dentists.

- Select the arch to be used, according to the strength and tooth movement desired.
- Fit the arch to the brackets and tubes glued to the tooth.
- Tie the arch with a metal ligature, elastic band or close the bracket clip.
- Cut the leftover bow using a suitable tool.

NOTE: Aditek Arch Sequences are used exclusively inside the oral cavity, being positioned on brackets and tubes glued to the teeth.

This product was developed exclusively for application in orthodontic treatment and for human use.

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