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1. Product Name:

1.1. Technical Name: Acid Conditioners

1.2. Commercial Name: Acid Conditioner Gel

1.3. Commercial Model: 1 - LightBond Gel Conditioner - 1 polypropylene syringe containing 2ml of phosphoric acid 37% (gel); 2 - LightBond Gel Conditioner - 1 polypropylene bottle containing 2ml of phosphoric acid 37% (gel); 3 - LightBond Gel Conditioner - 1 polypropylene syringe containing 2.5ml of 37% phosphoric acid (gel); 4 - LightBond Gel Conditioner - 1 polypropylene bottle containing 2.5ml of phosphoric acid 37% (gel); 5 - LightBond Gel Conditioner - 1 polypropylene syringe containing 3ml of phosphoric acid 37% (gel); 6 - LightBond Gel Conditioner - 1 polypropylene bottle containing 3ml of phosphoric acid 37% (gel); 7 - LightBond Gel Conditioner - 1 syringe of polypropylene containing 4ml of phosphoric acid 37% (gel); 8 - LightBond Gel Conditioner - 1 polypropylene bottle containing 4ml of phosphoric acid 37% (gel); 9 - LightBond Gel Conditioner - 1 polypropylene syringe containing 5ml of phosphoric acid 37% (gel); 10 - LightBond Gel Conditioner - 1 polypropylene bottle containing 5ml of phosphoric acid 37% (gel); 11 - LightBond Gel Conditioner - 1 polypropylene syringe containing 6ml of phosphoric acid 37% (gel); 12 - LightBond Conditioning Gel - 1 polypropylene bottle containing 6ml of 37% phosphoric acid (gel); 13 - LightBond Conditioning Gel - 1 polypropylene syringe containing 7ml of 37% phosphoric acid (gel); 14 - LightBond Gel Conditioner - 1 polypropylene bottle containing 7ml of 37% phosphoric acid (gel); 15 - LightBond Gel Conditioner - 1 polypropylene syringe containing 8ml of 37% phosphoric acid (gel); 16 - LightBond Gel Conditioner - 1 polypropylene bottle containing 8ml of 37% phosphoric acid (gel); 17 - LightBond Gel Conditioner - 1 polypropylene syringe containing 9ml of phosphoric acid 37% (gel); 18 - LightBond Gel Conditioner - 1 polypropylene bottle containing 9ml of phosphoric acid 37% (gel); 19 - LightBond Gel Conditioner - 1 polypropylene syringe containing 10ml of phosphoric acid 37% (gel); 20 - LightBond Gel Conditioner - 1 polypropylene bottle containing 10ml of phosphoric acid 37% (gel); 21 - LightBond Gel Conditioner - 1 polypropylene syringe containing 11ml of phosphoric acid 37% (gel); 22 - LightBond Gel Conditioner - 1 polypropylene bottle containing 11ml of phosphoric acid 37% (gel); 23 - LightBond Gel Conditioner - 1 polypropylene syringe containing 12ml of phosphoric acid 37% (gel); 24 - LightBond Gel Conditioner - 1 polypropylene bottle containing 12ml of phosphoric acid 37% (gel); 25 - Aditek Gel Conditioner - 1 polypropylene syringe containing 2ml of phosphoric acid 37% (gel); 26 - Aditek Gel Conditioner - 1 polypropylene bottle containing 2ml of acid phosphoric acid 37% (gel); 27 - Aditek Gel Conditioner - 1 polypropylene syringe containing 2.5ml of phosphoric acid 37% (gel); 28 - Aditek Gel Conditioner - 1 polypropylene bottle containing 2.5ml of phosphoric acid 37% (gel); 29 - Aditek Gel Conditioner - 1 polypropylene syringe containing 3ml of phosphoric acid 37% (gel); 30 - Aditek Gel Conditioner - 1 polypropylene bottle containing 3ml of phosphoric acid 37% (gel); 31 - Aditek Gel Conditioner - 1 polypropylene syringe containing 4ml of phosphoric acid 37% (gel); 32 - Aditek Gel Conditioner - 1 polypropylene bottle containing 4ml of phosphoric acid 37% (gel); 33 - Aditek Gel Conditioner - 1 polypropylene syringe containing 5ml of 37% phosphoric acid (gel); 34 - Aditek Gel Condition dispenser - 1 polypropylene bottle containing 5ml of phosphoric acid 37% (gel); 35 - Aditek Gel Conditioner - 1 polypropylene syringe containing 6ml of phosphoric acid 37% (gel); 36 - Aditek Gel Conditioner - 1 polypropylene bottle containing 6ml of phosphoric acid 37% (gel); 37 - Aditek Gel Conditioner - 1 polypropylene syringe containing 7ml of phosphoric acid 37% (gel); 38 - Aditek Gel Conditioner - 1 polypropylene bottle containing 7ml of phosphoric acid 37% (gel); 39 - Aditek Gel Conditioner - 1 polypropylene syringe

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containing 8ml of phosphoric acid 37% (gel);40 - Aditek Gel Conditioner - 1 polypropylene bottle containing 8ml of phosphoric acid 37% (gel);41 - Aditek Gel Conditioner - 1 polypropylene syringe containing 9ml of phosphoric acid 37% (gel);42 - Aditek Gel Conditioner - 1 polypropylene bottle containing 9ml of phosphoric acid 37% (gel);43 - Aditek Gel Conditioner - 1 polypropylene syringe containing 10ml of phosphoric acid 37% (gel);44 - Aditek Gel Conditioner - 1 polypropylene bottle containing 10ml of 37% phosphoric acid (gel);45 - Aditek Gel Conditioner - 1 polypropylene syringe containing 11ml of 37% phosphoric acid (gel);46 - Aditek Gel Conditioner - 1 polypropylene bottle containing 11ml of phosphoric acid 37% (gel);47 - Aditek Gel Conditioner - 1 polypropylene syringe containing 12ml of phosphoric acid 37% (gel);48 - Aditek Gel Conditioner - 1 polypropylene bottle containing 12ml of phosphoric acid 37% (gel).

1.4. Anvisa notification : 10331430040

2. Manufacturer:

Aditek do Brasil S.A .

Address: Rua Cesário Motta, 14

CEP: 14140-000 - Cravinhos – SP - Brazil

Phone: +55 (16) 3951-9355 / 0800- 0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Arantes Nogueira Neto - CREA: 601037885

www.aditek.com.br

3. Certifying Entity:

Not applicable.

4. Indication of Use:

The phosphoric acid present in the formulation, when in contact with dental tissues (enamel and dentin), carries out a chemical attack on the inorganic part, increasing the surface area of the tooth, and consequently the points of mechanical anchorage in adhesion processes. It is a gel with 37% phosphoric acid for the preparation of dental tissue (dentin and enamel) for adhesion of restorative materials for exclusive use in dental offices.

NOTE: The choice of material to be used is made by the qualified professional, based on his knowledge acquired during his training together with the diagnosis and treatment plan for each patient.

5. Users:

The use of the conditioning gel is done exclusively in a dental office by a trained professional.

6. Performance Characteristics:

Easy application for dental tissue preparation

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7. Benefit:

The Acid Conditioner Gel has the clinical benefit, through its action, of increasing the surface area of the tooth for mechanical anchorage in adhesion processes.

8. Warnings:

 Aditek products are designed and manufactured for single use only. Its reuse or reprocessing can cause cross-infection, decrease in bonding efficiency and loss of properties;

 If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products;

The professional must correctly follow the instructions for use;

The use environment must be aseptic;

Do not allow the product to come into contact with the eyes and skin;

If it comes into contact with eyes or skin, wash with plenty of water and do not rub. See a doctor immediately;

The product must be transported and kept in its closed packaging and protected from sunlight;

Store the product at temperatures between 5°C and 28°C and in humidity up to 75%;

Product for professional use only, keep away from unqualified people;

Keep out of the reach of children;

Do not use beyond the expiration date;

The professional must use good clinical practices for the disposal of medical products in accordance with current health standards;

Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use;

If serious incidents occur with the product, Aditek do Brasil must be notified to the competent authority in the national scope where the dentist is established;

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NOTE : Aditek do Brasil S.A. supplies products only to dental professionals. It is the dentist's sole responsibility to identify and/or possible contraindications that may prevent the use of the product.

10. Residual Risk:

Not applicable.

11. Undesirable Side Effects:

Not applicable.

12. Composition:

Components
Phosphoric Acid 85% GA
Polyethylene glycol 400
Blue Dye
Glycerol (Vegetable)
Nanoparticulate Silica
Distilled water

13. Shelf Life:

24 months.

14. Symbols and Meanings:

	Manufacturer		Do not reuse the product
	Consult instructions for use		Lot number
	Attention		Shelf Life
	Do not use if package is damaged or tampered with.		Product reference code
	Protect from light		Temperature limit

15. Information to Patient:

Avoid contact with eyes and skin, wash them immediately after contact and seek medical attention immediately;

The professional must correctly follow the instructions for use of the product. This product should only be used on patients by a dentist;

In case of ingestion or aspiration, consult a doctor.

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16. Mode of Use:

For the correct use of the product and obtaining satisfactory results in orthodontic treatment, carefully follow the steps described below:

- Prepare the region to be conditioned with pumice stone or oil -free paste (Clean and dry);
- Apply the acid conditioner to the surface for at least 20 seconds (some adhesive products have an application time defined by the manufacturer);
- After application remove the product using abundant water;
- Dry the spot to start the bonding procedure.

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Ácido fosfórico 85% AG