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1. Product name:

- 1.1. Technical Name:**Components for Orthodontics
- 1.2. Commercial name:** Aditek Aesthetic Brackets and Accessories
- 1.3. Commercial Model:** Aesthetic Distalizer
- 1.4. Anvisa Notification:**10331439003

2. Manufacturer:

Aditek do Brasil SA
Address: Cesario Mota, 14
Zip Code: 14140-000 - Cravinhos - SP - Brazil
Phone: +55 (16) 3951-9355 / 0800-0159520
CNPJ 64.602.097/0001-95
Technical Responsible: José Vicente Junior - CREA: 5069373070
www.aditek.com.br

3. Certifying Body:

Not applicable.

4. Indication of Use:

Aesthetic Distalizers are indicated as aids in orthodontic treatment. Its purpose of the use is to distalize or mesialize the upper premolars, molars, and canines, as well as to further improve patient acceptance and collaboration, as it is a more discreet and non-invasive product.

The products were developed to be used in the oral cavity environment, directly on the teeth, with the aim of supporting orthodontic treatments in humans, promoting a better positioning of the teeth within the dental arches, and seeking a balanced occlusion. They are indicated to treat patients who have malocclusion problems that require alignment and leveling movements of the teeth in the arches. They are chosen by orthodontic professionals who seek a more aesthetic clinical care.

NOTE: The choice of material to be used is made by the qualified professional, based on their knowledge acquired during their training together with the diagnosis and treatment plan for each patient.

5. Users:

The application of the Aditek Aesthetic Distalizer is restricted to dental professionals. The use by people outside this context may cause harm to the patient.

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6. Performance Characteristics:

Commonly, an orthodontic treatment without interruption takes 6 to 8 months, varying in each case according to the facial pattern, type of malocclusion and oral and systemic conditions of the patient, which may be extended according to the complexity of the case.

7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are the aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

8. Warnings:

 Aditek products are designed and manufactured for single use only. Its reuse or reprocessing can cause cross-infection, decrease in efficiency and loss of mechanical properties due to natural wear and tear and forces applied when removing the product.

 If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

Store the product in a cool, dry place, away from sunlight, keeping it in its own packaging until use.

The product must be applied by a professional orthodontist who must follow the instructions for use and guidelines specified by the manufacturer.

The product is safe for use in pregnant women. It is known that due to hormonal changes during pregnancy, women are more susceptible to periodontal problems. During orthodontic treatment, the difficulty of cleaning can aggravate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

In the treatment and disposal of the product, its remains and used packaging, attention must be paid to local, state and national legislation.

The composition of the product does not require specific procedures for its transport and does not pose a risk to people. Protect from moisture.

The product is supplied in non-sterile condition, however it has proven biocompatibility and cleaning, disinfection or sterilization measures are not necessary, leaving it up to the professional to carry out the sterilization.

The professional must use good clinical practices for the disposal of medical products in accordance with current health standards.

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Aditek do Brasil SA is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

If serious incidents occur with the product, Aditek do Brasil SA must be notified to the competent authority within the national scope where the dentist is established.

9. Contraindications:

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;
- Illness or limitation that may make treatment difficult;
- Existing bone or root resorption;
- Decalcification of existing tooth enamel;
- Existing periodontal complications.

NOTE: Aditek do Brasil SA supplies products only to dental professionals. It is the dentist's sole responsibility to identify and/or possible contraindications that may prevent the use of the product.

10. Residual Risks:

All finished orthodontic treatments are subject to relapse. After treatment, control must be determined by the professional in order to avoid or minimize recurrence.

11. Undesirable Side-Effects:

The use of the medical device can lead to discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort and pain.

12. Composition:

polycarbonate
Chemical composition limits (%)
100

Contains no hazardous ingredients as per Regulation (EC) No. 1907/2006

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14. Symbols and Meanings:

	Manufacturer		Lot number
	Consult instructions for use		Product reference code
	Attention		Expiration date
	Do not use if package is damaged or tampered with.		Do not reuse the product

15. Information to Patient:

The patient must carry out good daily hygiene in order to avoid the installation of bacterial plaque and consequent periodontal diseases and to remove food particles that may settle in the aesthetic distalizer.

The use of an oral irrigator, Water Pik type, is recommended to help clean teeth and aesthetic distalizers.

It is important that the pregnant patient is referred for professional cleaning at regular intervals, due to the increased susceptibility of periodontal problems during pregnancy.

It is important to be careful when chewing certain types of food, especially those that are harder or crunchier, as they can detach or break the distalizers. These foods must be cut before eating. Hard foods like popcorn and ice should be avoided.

Do not put objects in the mouth, such as pens and pencils, for example, and be careful not to hit or bite the structure of the brush when brushing your teeth, which could break or loosen the aesthetic distalizer when subjected to a lever movement.

Smoking or chewing tobacco can stain your esthetic distalizers.

In case of damage, consult an orthodontist.

In case of ingestion or aspiration, consult a doctor.

When participating in sports, always wear mouthguards, as impacts can damage the braces and injure the oral cavity.





The patient must follow the professional's guidelines regarding containment to avoid recurrence.

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
16. Mode of Use:

The steps below should only be performed by dental professionals:

1. Glue the tube on the molar;
2. Measure the distance from the tube entrance to the long axis of the clinical crown of the Canine;
3. Add 6mm to the measured measurement;
4. Cut the posterior excess of the arm flush with the ruler with a string cutter;
5. 6mm corresponds to the measurement from the entrance to the bottom of the tube;
6. Fit the distalizer into the tube and bond it to the tooth.

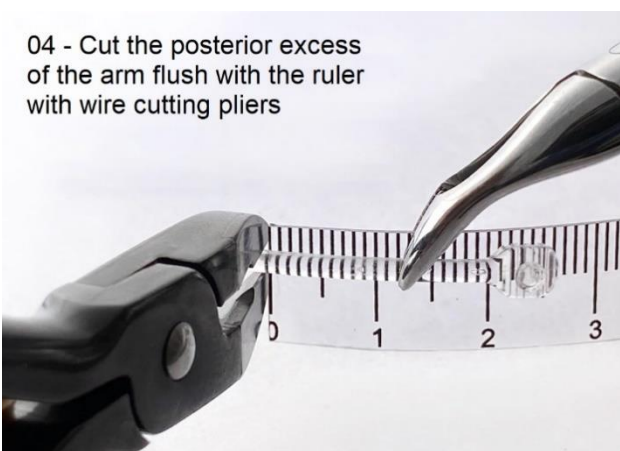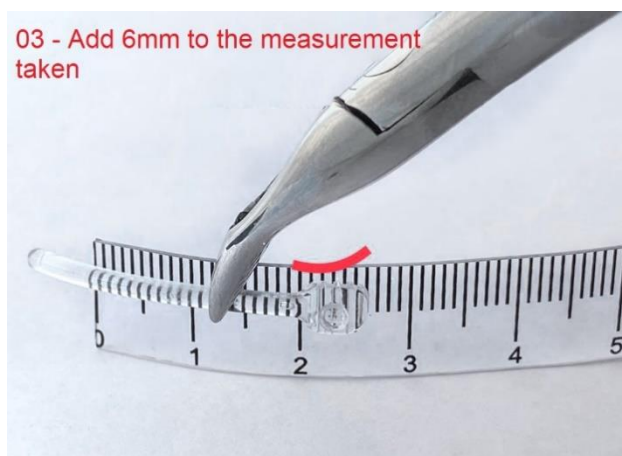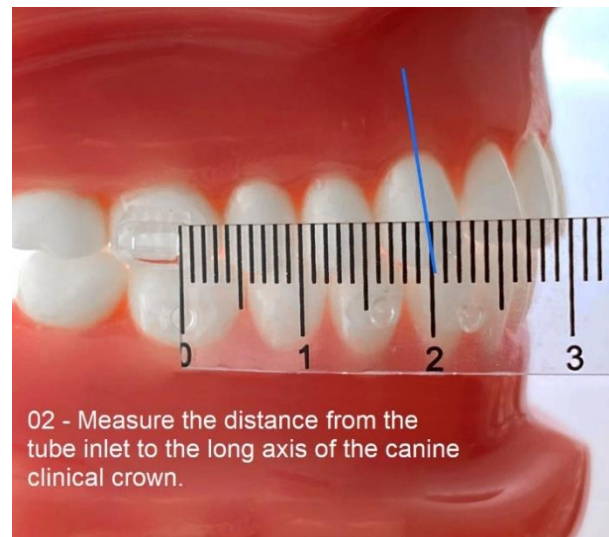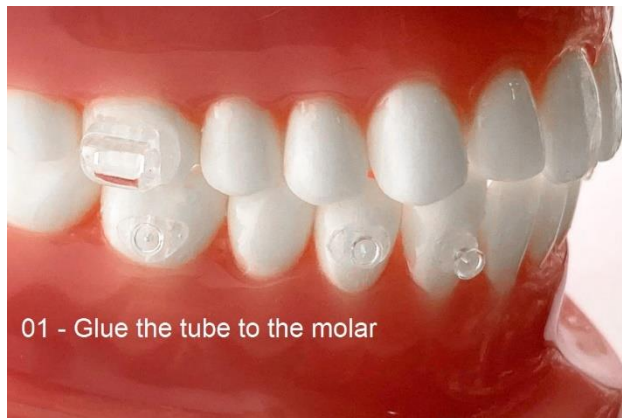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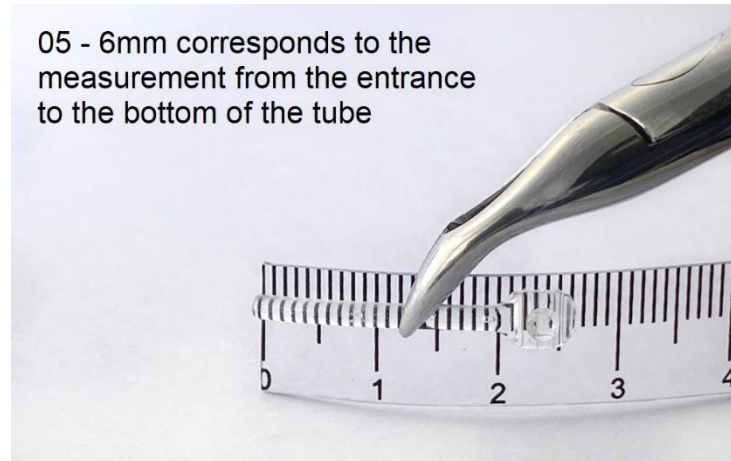NOTE: Aditek Esthetic Distalizers and Accessories must be glued directly onto the surface of the teeth by its base, using orthodontic adhesives, following the adhesive manufacturer's instructions for use (we suggest using LightBond - Aditek adhesive).

16.1. Removing the Distalizer and Aesthetic Accessory:

- Remove resin around the distalizer and/or accessory with a high-speed resin removal bur.
- Position the removal instrument mesiodistally and move clockwise.
- The distalizer and/or accessory should come off, leaving a small portion of resin on the tooth, which should be removed with a high-speed bur for resin removal or resin removal pliers.

NOTE: If you have difficulty removing the distalizer, use a diamond cutter to section the distalizer horizontally, at the height of the middle of the piece, taking care not to touch the tooth enamel. Once sectioned, remove the parts with a suitable instrument.

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