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1. Product name:

1.1. Technical Name: Device for moving teeth

1.2. Commercial name: Removable Aesthetic Aligners

1.3. Anvisa Notification: 10331439002

2. Manufacturer:

Aditek do Brasil SA

Address: Cesario Mota street, 14.

Zip Code: 14140-000 - Cravinhos - SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical responsible: José Vicente Junior - CREA: 5069373070

www.aditek.com.br

www.aditekortho.com

3. Certifying Body:

Not applicable.

4. Indication of Use:

Removable Aesthetic Aligners are made up of transparent thermoplastic trays that are fitted to the upper and lower dental arches, with the function of promoting strictly dental (non-skeletal) movement gradually in orthodontic treatment, respecting the dentist's evaluation and the limitation of tooth movement of maximum 4mm.

They are indicated for patients with mixed and permanent dentition.

NOTE: The choice of material to be used is made by the qualified professional, based on the knowledge acquired during their training together with the diagnosis and treatment plan for each patient.

5. Users:

The application of the Removable Aesthetic Aligner is restricted to dental professionals with training in Orthodontics. The use by people outside this context may result in harm to the patient.

6. Performance Characteristics:

Commonly, uninterrupted orthodontic treatment takes 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion, and oral and systemic conditions of the patient, which may extend according to the complexity of the case.

Removable Aesthetic Aligners should be changed every 1 to 2 weeks, according to the dentist's instructions. Using it for a time other than that recommended may lead to the ineffectiveness of the treatment. Once positioned in the patient's upper and lower arches, the aligners provide specific forces to carry out the dental movements predicted in the virtual treatment, correcting the patient's dentition and occlusion.

Each Removable Aesthetic Aligner is unique for each patient and in case of loss, a new one must be requested for replacement.

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7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, and better speech and breathing.

8. Warnings:

Aditek products are designed and manufactured for single use only. Its reuse or reprocessing may cause cross-infection.

If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

Store the product in a clean, cool, dry place and away from sunlight, keeping it in its own packaging until use, so as not to cause changes in its shape.

The product is safe for use in pregnant women. It is known that due to hormonal changes during pregnancy, women are more susceptible to periodontal problems. During orthodontic treatment, the difficulty of cleaning can aggravate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

The composition of the product does not require specific procedures for its transport and does not pose a risk to people.

The product is supplied in non-sterile condition; however it has proven biocompatibility and no cleaning, disinfection or sterilization measures are required.

It is recommended that the patient strictly follow the instructions for use given by their orthodontist and in case of any doubt, the patient should contact their dentist as soon as possible.

The professional must use good clinical practices for the disposal of medical products in accordance with current health standards.

Aditek do Brasil SA is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

If serious incidents occur with the product, Aditek do Brasil must be notified to the competent authority at the national level where the orthodontist is established.

9. Contraindications:

The Removable Aesthetic Aligner system is contraindicated for:

- Patients who are allergic to copolyester;
- Patients who are allergic to silicone, in the case of using the Aligner Teether;

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- Patients with caries and severe periodontal problems;
- Patients who have many tooth losses;
- Patients who have previous temporomandibular disorders;
- Patients who do not fit the malocclusion pattern of the aligner system.

NOTE: The orthodontist is entirely responsible for identifying and/or possible contraindications that may prevent the use of the product.

10. Residual Risks:

All finished orthodontic treatments are subject to relapse. After treatment, control must be determined by the professional in order to avoid or minimize recurrence.

11. Undesirable Side-Effects:

As a secondary effect, at the end of the treatment, a slight posterior open bite is often observed due to the thickness of the aligner, which is easily corrected after the end of the treatment.

The aligners are made of highly resistant polymer with reduced thickness, in order to avoid discomfort with their use. Therefore, the patient should handle them gently, not bending or twisting them, as they could be permanently damaged.

The patient should not change the aligners or move on to the next stages of treatment without prior guidance from the dentist. This may cause ineffective treatment.

It is recommended that the patient strictly follow the instructions for use and in case of any doubt, he should contact his dentist as soon as possible.

12. Composition:

COPOLYESTER
Chemical composition limits (%)
100%

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14. Symbols and Meanings:

	Manufacturer		Product reference code
	Consult instructions for use		Expiration date
	Attention		Do not reuse the product
	Do not use if package is damaged or tampered with.		Lot number
	Keep away from sunlight		

15. Patient Information:

Use the aligner as much as possible for the success of your treatment, it is recommended 22 hours a day, removing it only for food and hygiene;

Speech may be altered in the first few hours, but depending on its use, the patient will adapt;

Some pain, sensitivity is expected due to tooth movement, but at a lower intensity than would be the case with conventional fixed appliances. If the pain is higher than expected, contact your dentist and eventually stop using it;

If the edge of the aligner is causing discomfort, the patient can ask the orthodontist to polish the site using the sandpaper that comes with the kit;

Avoid eating sticky candies, chewing gum and smoking cigarettes at the risk of damaging the aligners;

Hot drinks that contain dyes should not be ingested, as they can deform the plates and change the color of the aligners;

Keep the aligners in specific cases when out of the mouth, for your protection;

Keep aligners always sanitized and clean. To clean the aligners, brushing with paste is recommended. no abrasive and toothbrushes soft-bristled.

Always perform oral hygiene before using the aligners.

16. Mode of Use:

- Insert the aligner into the dental arches, fit first the anterior part of the front teeth, and then the posterior part, always carefully avoiding sudden movements;

NOTE: Do not bite the aligner to fit, always use the teether for aligner, which also comes with your kit. The rollers must be bitten with both aligners positioned in the arches, posterior and anterior, a few times, in the first two days after changing the plates.

- Wear your aligner as long as possible, considering that the recommendation for use is 22 hours a day, removing it only for feeding and cleaning. In general, exchanges are at two-week intervals, unless your dentist advises you differently;
- To remove the aligner, always unhook the lingual and palatal part first using the “hook” (Solt), which comes with your kit, and then the anterior part;

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- After removing the aligner, wash it thoroughly under running water or with a cleaning foam, and always keep it in the case provided, avoiding leaving it in hot environments or exposed to the sun.

NOTE: Store the anterior aligner after replacing it with a new one. If you lose your current plate, use the previous one and contact your orthodontist.

16.1. How to Use the Accessories:

- **Solt:** It has an end to help remove the aligners and the other end has a ring to facilitate support. Sanitize after use.
- **Aligner Teether:** After placing the aligners on the teeth, place the rollers (Aligner Teether) between the back teeth and bite until the aligners are fully seated. Sanitize after use.
- **Sandpaper:** Eventually, the patient may feel a slight discomfort at the edges of the aligner, in which case the dentist uses the sandpaper to trim the area that is bothering.

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