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1. Product name:

1.1. Technical Name:ORTHODONTIC ELASTICS

1.2. Commercial Name:ADITEK ORTHODONTIC ELASTICS

1.3. Commercial Model: SEPARATORS – 4.00 mm (5/32”) and 4.80mm (3/16”) in diameter; Chain elastic: ELASTIC CHAIN (Elastik): continuous, medium and long; Rolls, 1m / 1.5m / 2m / 2.5m / 3m / 3.5m / 4m / 4.5m / 5m / 5.5m / 6m / 6.5m / 7m / 7.5m / 8m / 8.5m / 9m / 9.5m and 10m; ELASTIC LIGATURE (Stick Tie)

1.4. Anvisa Notification:10331430012

2. Manufacturer:

Aditek do Brasil SA

Address: Cesario Mota street, 14.

Zip Code: 14140-000 - Cravinhos - SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Vicente Junior - CREA: 5069373070

www.aditek.com.br

3. Certifying Body:

Not applicable.

4. Indication of Use:

Elastics are indicated to assist orthodontic treatments. Its application is performed following a pre-established treatment by a professional in the field of dentistry. The purpose of rubber bands is to provide traction between the points at which they are fixed. The force with which the rubber bands are pulled is what will modify and correct the position of the teeth. Another purpose of rubber bands is to attach wires and/or bows to the Brackets. The wire that passes through the Bracket slot is fixed by the elastic that is hooked to the Bracket fins, preventing the wire from moving and leaving its position.

NOTE: The choice of material to be used is made by the qualified professional, based on the knowledge acquired during their training together with the diagnosis and treatment plan for each patient.

5. Users:

The application of Aditek Orthodontic Elastics is restricted to dental professionals. The use by people outside this context may result in irreversible damage to the patient.

6. Performance Characteristics:

Aditek orthodontic elastics exhibit high deformation under load, returning to their original shape after this load has been relieved. They are used as accessories, conjugates, or coupled to

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orthodontic devices, which serve as support for the release of a force, generated by its own elastic property, moving and correcting teeth during treatment, aiding tooth movement, and correction of the arches. They are manufactured in various diameters, thus enabling a varied intensity of force correction of the arches.

7. **Clinical Benefits:**

The main benefits that these systems bring to the patient's life are an aesthetic improvement with the alignment and leveling of teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

8. **Warnings:**

 Aditek products are designed and manufactured for single use only. Its reuse or reprocessing can cause cross-infection and a decrease in efficiency.

 If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

Store the product in a cool place with a temperature between 15° and 30°C, dry and away from sunlight, and it can be packed in its own packaging.

This product may cause allergic reactions in patients susceptible to allergies to substances that make up Aditek Orthodontic Elastics. In case of reaction, remove the elastic immediately and advise the patient to see a doctor.

This product does not require special transport procedures. Can be transported in its own packaging.

The product is supplied in non-sterile condition.

The professional must use good clinical practices for the disposal of medical products in accordance with current health standards.

Aditek do Brasil SA is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

If serious incidents occur with the product, Aditek do Brasil and the competent authority at the national level must be notified.

9. **Contraindications:**

- Patients with poor oral hygiene;
- Patients unable to cooperate with treatment;

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- Illness or limitation that may make treatment difficult;
- Existing bone or root resorption;
- Decalcification of existing tooth enamel;
- Existing periodontal complications;
- Patients with hypersensitivity to polyurethane.

The orthodontist is entirely responsible for identifying and/or possible contraindications that may prevent the use of the product.

10. Residual Risks:

All finished orthodontic treatments are subject to relapse. After treatment, control must be determined by the professional to avoid or minimize recurrence.

11. Undesirable Side Effects:

The use of the medical device can lead to allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort, and pain.

12. Composition:

Nontoxic Polyurethane
Chemical composition limits (%)
100%

13. Shelf life:

 Undetermined.

14. Symbols and Meanings:

	Manufacturer		Product reference code
	Consult instructions for use		Expiration date
	Attention		Do not reuse the product
	Do not use if package is damaged or tampered with.		Lot number
	Keep away from sunlight		

15. Patient Information:

Non-toxic product;
 Perform good daily hygiene in order to avoid the installation of bacterial plaque and consequent periodontal diseases;

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It is important that the pregnant patient is referred for professional cleaning at regular intervals, due to the increased susceptibility of periodontal problems during pregnancy;

Chewing hard foods can cause braces to break or fall off;

In case of damage, consult an orthodontist;

In case of ingestion or aspiration, consult a doctor;

When participating in sports, always use mouthguards, as impacts can damage the appliances and hurt the oral cavity;

The patient must follow the professional's guidelines regarding containment to avoid recurrence.

16. Mode of use:

Aditek Orthodontic Elastics are fixed and positioned in other devices used in orthodontic treatment such as brackets, tubes, and buttons, among others. Orthodontic elastics must be positioned on the hooks and fins between two brackets, tubes, and/or buttons, thus providing the necessary force for the movement and correction of the teeth.

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