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## 1. Product's name:

**1.1. Technical Name:** Device for moving teeth

**1.2. Commercial name:** Removable Cosmetic Aligners

**1.3. Anvisa/MS Notification:** 10331430041

## 2. Manufacturer:

Aditek do Brasil SA

Address: Cesario Mota, 14.

CEP: 14140-000 - Cravinhos - SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Vicente Junior - CREA: 5069373070

[www.aditek.com.br](http://www.aditek.com.br)

[www.aditekortho.com](http://www.aditekortho.com)

## 3. Certifying Body:

Not applicable.

## 4. Indication of Use:

Removable Esthetic Aligners are made up of transparent thermoplastic trays that are fitted to the upper and lower dental arches, with the function of promoting strictly dental movement (not skeletal) gradually in orthodontic treatment, respecting the evaluation of the orthodontist and the limitation of tooth movement of maximum 4mm. They are indicated for patients with mixed and permanent dentition.

**NOTE:** The choice of material to be used is made by the qualified professional, based on their knowledge acquired during their training together with the diagnosis and treatment plan for each patient.

## 5. Users:

The application of the Removable Esthetic Aligner is restricted to dental professionals with training in Orthodontics. The use by people outside this context may cause harm to the patient.

## 6. Performance Features:

Commonly, uninterrupted orthodontic treatment takes 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion and oral and systemic conditions of the patient, which may extend according to the complexity of the case.

Removable Esthetic Aligners should be changed every 1 to 2 weeks, according to the orthodontist's guidance. Using it for a time other than that recommended may lead to ineffectiveness of the

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treatment. Once positioned in the patient's upper and lower arches, the aligners provide specific forces to carry out the dental movements predicted in the virtual treatment, correcting the patient's dentition and occlusion.

## 7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are the aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient masticatory function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

## 8. Warnings:

 Aditek products are designed and manufactured for single use only. Its reuse or reprocessing may cause cross-infection.

 If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible falsification of products.

 Store the product in a clean, cool, dry place and away from sunlight, keeping it in its own packaging until use, so as not to cause changes in its shape.

The composition of the product does not require specific procedures for its transport and does not pose a risk to people.

The product is supplied in non-sterile condition, requiring cleaning before use.

It is recommended that the patient strictly follow the instructions for use given by his orthodontist and in case of any doubt, the patient should contact his orthodontist as soon as possible.

The professional must use good clinical practices for the disposal of medical products in accordance with current health standards.

Aditek do Brasil SA is not responsible for the results obtained by applying the product without proper precautions or failure to observe the warnings and instructions for use.

If serious incidents occur with the product, Aditek do Brasil SA must be notified to the competent authority within the national scope where the orthodontist is established.

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## 9. Contraindications:

The Removable Cosmetic Aligner system is contraindicated for:

- Patients who are allergic to polyurethane.
- Patients with caries and severe periodontal problems.
- Patient who have many tooth losses.
- Patients who have previous temporomandibular disorders.
- Patients who do not fit the malocclusion pattern of the aligner system.

**NOTE:**The orthodontist is entirely responsible for identifying and/or possible contraindications that may prevent the use of the product.

## 10. Residual Risks:

All finished orthodontic treatments are subject to relapse. After treatment, control must be determined by the professional in order to avoid or minimize recurrence.

## 11. Undesirable Side Effects:

As a secondary effect, at the end of the treatment, a slight posterior open bite is often observed due to the thickness of the aligner, which is easily corrected after the end of the treatment.

The patient may find it strange to talk with the aligner in their mouth, but as they use it, they will easily get used to it.

Some sensitivity is expected due to tooth movement, but at a lower intensity than would be the case with a conventional fixed appliance.

## 12. Composition:

<b>Thermoplastic Polyurethane (TPU)</b>
Chemical composition limits (%)
100%

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#### 14. Symbols and Meanings:

	Manufacturer		Lot number
	Consult instructions for use		Expiration date
	Attention		Keep away from sunlight
	Do not use if package is damaged or tampered with.		Do not reuse the product
	Product reference code		

#### 15. Patient Information:

The patient must use the aligner as much as possible for the success of his treatment, it is recommended 22 hours a day, removing it only for feeding and cleaning.

Speech may be altered in the first few hours, but depending on its use, the patient will adapt.

Some pain, sensitivity is expected due to tooth movement, but at a lower intensity than would be the case with conventional fixed appliances, if the pain is one degree higher than expected, the patient should immediately contact their orthodontist and follow their instructions. recommendations.

If the edge of the aligner is causing discomfort, the patient can ask the orthodontist to polish the area using the sandpaper that comes with the kit.

At the risk of damaging the aligners, it is not recommended for the patient to consume sticky candies, chewing gum and smoke cigarettes.

Hot drinks that contain dyes should not be ingested, as they can cause deformation and change the color of the aligners.

Keep the aligners in specific cases when out of the mouth, for your protection.

Keep aligners always sanitized and clean. For cleaning, brushing with a non-abrasive paste and toothbrushes with soft bristles is recommended.

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The patient should not use hot water on the aligners.

The patient should not change the aligners or move on to the next stages of treatment without prior guidance from the orthodontist. This may cause the treatment to be ineffective.

The aligners are made of highly resistant polymer with reduced thickness, in order to avoid discomfort with their use. Therefore, the patient should handle them gently, not bending or twisting them, as there is a risk of permanent damage to them.

Each Removable Esthetic Aligner is unique for each patient and in case of loss or breakage, the patient must inform their orthodontist to request a new one.

If, even after using the aligner teether, the aligner does not adapt to the dental arch, the patient should contact their orthodontist.

## 16. Mode of Use:

Carefully read the instructions below and, if in doubt, contact your orthodontist before using the aligners.

### Inserting the Aligner

- Perform oral hygiene.
- Clean the aligner with a soft bristle brush and non-abrasive toothpaste.
- Before placing the aligner, make sure you have the correct aligner.
- With clean hands, insert the aligner into the dental arch, fitting first the anterior part, the front teeth, and then the posterior part, always carefully avoiding sudden movements.
- After fitting the upper and lower aligners, position the aligner teeth between the posterior teeth, located at the bottom of the dental arch, and perform gentle chewing movements.

**NOTE:** Do not bite the aligner to fit, always use the aligner teether, which also comes with your kit. The rollers (bites) must be gently bitten with both aligners, upper and lower, positioned in the dental arch, sometimes, in the first two days after changing the plates.

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- To remove the aligner, always carefully detach first the lingual and palatal part using the “hook” (Solt), which comes with your kit, and then the anterior part.
- After removing the aligner, wash it thoroughly under running water, or with a soft bristle brush and non-abrasive toothpaste, and always store it in the aligner holder provided, avoiding leaving it in hot environments or exposed to the sun.

**NOTE:**Store the anterior aligner after replacing it with a new one. If you lose your current plate, use the previous one and contact your orthodontist.

- Wear your aligner as long as possible, considering that the recommendation for use is 22 hours a day, removing it only for feeding and cleaning. In general, changes are made at two-week intervals, unless your orthodontist instructs you otherwise.

#### 16.1. How to Use the Accessories:

- **Release:**It has one end to help remove the aligners and the other end has a ring where the patient can fit the finger to facilitate support. Sanitize after use.
- **Teether for Orthodontic Aligner:**After fitting the aligners on the teeth, place the teethers between the posterior teeth and gently bite until the aligners are fully seated. Sanitize after use.
- **Sandpaper:**Eventually, the patient may feel a slight discomfort at the edges of the aligner, in which case the orthodontist may use sandpaper to polish the area that is bothering him.
- **Aligner Port:**When not using the aligner, always store it in the aligner holder, in order to prevent damage to the aligner. Keep the aligner holder sanitized.

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