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1. Product' Name:

1.1. Technical Name: Dental Acrylic Resins.

1.2. Commercial Name: Horus BioPrint.

1.3. Commercial Model: Horus BioPrint Guide; Horus BioPrint IBT; Horus BioPrint Splint Soft; Horus BioPrint Splint Hard.

1.4. Anvisa Notification: 10331439007

2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Mota Street, 14

Zip Code: 14140-000 - Cravinhos – SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Vicente Junior - CREA: 5069373070

www.aditek.com.br

www.aditekortho.com

3. Certifying Body:

Not applicable.

4. Indication of Use:

Horus BioPrint Dental Acrylic Resins are photoactivated resins curable by UV light, exclusive for use in 3D printers, which are suitable for the following applications:

- The Horus BioPrint Guide is suitable for the manufacture of surgical guides, orthognathic surgical guides, and implant positioning guides.
- Horus BioPrint IBT is suitable for making guides or trays for indirect bonding of Brackets.
- The Horus BioPrint Splint Soft and the Horus BioPrint Splint Hard are indicated for the manufacture of my refractive plaques or bruxism plaques.

5. Users:

The use of resin is restricted to qualified professionals. The use by people outside this context may cause damage to the user of the final product.

6. Performance Characteristics:

The working principle of Dental Acrylic Resins is given by the photopolymerization of oligomers and monomers directed by ultraviolet light in the 3D printer, obeying the format of the file previously established according to the patient's data and the use of each final product.

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Not applicable.

8. Warnings:

If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible counterfeiting of products.

The product must be transported in its original packaging, at room temperature, and low humidity, and not be exposed to the sun or UV light.

The product must be used by a qualified professional who must follow the instructions for use and guidelines specified by the manufacturer.

Must be stored and maintained at a temperature between 2°C and 28°C, with low humidity, and not be exposed to the sun or UV light.

Heavy objects should not be deposited on the product packaging.

The professional must use good clinical practices for the disposal of medical products according to current sanitary standards to avoid contamination of land, plumbing, and surface water.

The product can be toxic to aquatic organisms and can cause long-term adverse side effects on the aquatic environment (H413).

Aditek do Brasil S.A. is not responsible for the results obtained by the application of the product without due precautions or failure to observe the warnings and instructions for use.

Keep out of reach of children and animals.

Product supplied in non-sterile condition.

The product can be irritating to the skin, eyes, and respiratory system, mandatory use of Epis.

In the event of serious incidents with the product, Aditek do Brasil shall notify the competent national authority where the dentist is established.

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9. Contraindications:

The product does not have contraindications when used for its intended purpose, following its instructions for use.

It is the sole responsibility of the qualified professional to identify and/ or possible contraindications that may prevent the use of the product.

10. Residual Risks:

Not applicable.

11. Unwanted Side Effects:

Not applicable.

12. Composition:

Chemical Name	Weight Percentage (%)
Monomers and Acrylic Oligomers	> 96%
Pigments	≤ 1%
Photoinitiators	< 3%

The commercial models of the family are constituted of the same base of chemical components with variations of their concentrations to meet the specificity of each application.

13. Shelf life:

2 years after manufacture with sealed packaging. 3 months after opening the bottle, protected against light, humidity, and in an airy place.

14. Symbols and Meanings:

	Manufacturer		Product reference code
	Consult instructions for use		Lot number
	Attention		Keep away from sunlight
	Do not use if the package is damaged or violated		Shelf life

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15. Patient Information:

During product handling, it is mandatory to use personal protective equipment (Epi), such as gloves and glasses.

Shake the bottle well before using the product.

Should not be used if the expiry date is expired.

Its cleaning can be done with the use of isopropyl or ethyl alcohol, then with soap and water.

Keep the bottle closed after use. Store between 2°C and 28°C.

16. Mode of Use:

- The ideal working temperature of the resin is from 20°C to 31°C.
- The product should be handled in a UV-free environment, preferably in a room with warm yellow or white light.
- Use gloves, glasses, and apron or lab coat to manipulate the product.
- Check that the printer is calibrated with the print profile indicated by the manufacturer for the respective resin so that the printer can be adjusted and print correctly.
- The product must be shaken before use, to ensure the homogenization of its components.
- Must be placed carefully in the printer tank, respecting the volume indicated by the manufacturer of each printer.
- After printing, the product should be removed carefully from the printer bed, as it has not yet passed the post-curing and is still fragile.
- It should be washed with isopropyl alcohol or ethyl under mechanical agitation or cavitation (use of agitation with ultrasound), preferably in two consecutive baths, to ensure that there is no liquid resin in the final piece, each bath can be performed from 3 to 5 minutes, important not to exceed this time not to weaken the piece.
- The piece must be dried with the use of compressed air or by resting on a paper towel.
- If used in the process, print media in the file, must be removed after drying, removal at this stage is easier and does not weaken the final part.
- After complete drying, it must be placed in a UV light chamber, for the final post-curing of 30 to 90 min.
- After finishing the process, the liquid resin should be stored in your bottle, do not leave the resin in the printing tank.

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