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## 1. Product's Name:

**1.1. Technical Name:** Plate for Making of Trays.

**1.2. Commercial Name:** Memoflex TPU Thermoplastic Plate.

**1.3. Commercial Model:** 01 unit: 0,30 mm, 0,40 mm, 0,50 mm, 0,60 mm, 0,75 mm, 0,76mm, 0,80 mm e 1,00 mm; 03 units: 0,30 mm, 0,40 mm, 0,50 mm, 0,60 mm, 0,75 mm, 0,76mm, 0,80 mm e 1,00 mm; 05 units: 0,30 mm, 0,40 mm, 0,50 mm, 0,60 mm, 0,75 mm, 0,76mm, 0,80 mm e 1,00 mm; 10 units: 0,30 mm, 0,40 mm, 0,50 mm, 0,60 mm, 0,75 mm, 0,76mm, 0,80 mm e 1,00 mm; 20 units: 0,30 mm, 0,40 mm, 0,50 mm, 0,60 mm, 0,75 mm, 0,76mm, 0,80 mm e 1,00 mm; 25 units: 0,30 mm, 0,40 mm, 0,50 mm, 0,60 mm, 0,75 mm, 0,76mm, 0,80 mm e 1,00 mm; 30 units: 0,30 mm, 0,40 mm, 0,50 mm, 0,60 mm, 0,75 mm, 0,76mm, 0,80 mm e 1,00 mm; 40 units: 0,30 mm, 0,40 mm, 0,50 mm, 0,60 mm, 0,75 mm, 0,76mm, 0,80 mm e 1,00 mm; 50 units: 0,30 mm, 0,40 mm, 0,50 mm, 0,60 mm, 0,75 mm, 0,76mm, 0,80 mm e 1,00 mm e 100 units: 0,30 mm, 0,40 mm, 0,50 mm, 0,60 mm, 0,75 mm, 0,76mm, 0,80 mm e 1,00 mm.

## 1.4. Anvisa Notification:

10331430042

## 2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Mota Street, 14

Zip Code: 14140-000 - Cravinhos - SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Responsible: José Vicente Junior - CREA: 5069373070

[www.aditek.com.br](http://www.aditek.com.br)

[www.aditekortho.com](http://www.aditekortho.com)

## 3. Certifying Body:

Not applicable.

## 4. Indication of Use:

Memoflex plates are suitable for the manufacture of dental trays and according to their characteristics and thicknesses, for different dental applications such as orthodontic aligners, occlusion plates, trays for indirect bonding of Brackets, dental whitening plates and orthodontic retaining plates.

## 5. Users:

The use of plates for making trays is restricted to dental professionals. Use by people outside this context may cause harm to the patient. The user undertakes to verify his responsibility for the

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suitability of the product for the intended use, especially in the case of fields of use not expressly indicated in the instructions for use.

## 6. Performance Characteristics:

This product uses unique high-performance polyurethane materials for medical use, which combine high strength, flexibility, low-stress relaxation, and toughness of materials.

Memoflex has excellent transparency and color stability.

The plate is easily formed and can be created by a vacuum or pressure thermoforming machine.

## 7. Clinical Benefits:

Thermoformed polymer: Fidelity in the reproduction of the arches.

Excellent translucency: Aesthetic during use.

## 8. Warnings:

Aditek products are designed and manufactured for single use. Its reuse or reprocessing can cause cross-infection and decrease efficiency.

If the packaging has been violated, do not use the product, avoiding risks related to the transmission of microorganisms or the possible counterfeiting of products.

Store the product at room temperature, dry and away from sunlight, keeping it in its own packaging, so as not to cause changes in its shape.

This product should be transported in its own packaging, should not be folded or kneaded, and protected from moisture.

The product is supplied in non-sterile condition.

The professional must use good clinical practices for the disposal of medical products according to current sanitary standards.

Aditek do Brasil S.A. is not responsible for the results obtained by the application of the product without due precautions or failure to observe the warnings and instructions for use.

In the event of serious incidents with the product, Aditek do Brasil and the national competent authority shall be notified.

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Caution if the plate is heated for a very long period during thermoforming the material may soften presenting risks of burns. Use appropriate protective equipment.

#### 9. Contraindications:

The product is contraindicated for patients with an allergic reaction to the material.

#### 10. Residual Risks:

Not applicable.

#### 11. Unwanted Side Effects:

Not applicable.

#### 12. Composition:

<b>Thermoplastic Polyurethane (TPU)</b>
Chemical composition limits (%)
100%

#### 13. Shelf Life:

2 years.

#### 14. Symbols and Meanings:

	Manufacturer		Shelf life
	Consult instructions for use		Keep away from rain
	Attention		Keep away from sunlight
	Do not use if the package is damaged or violated		Do not reuse the product
	Product reference code		Lot number

#### 15. Patient Information:

When working with any model of the plate for making trays, during the making and finishing of the tray, always use appropriate personal protective equipment.

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The responsibility of clinical treatment, technical for the construction of the tray, side effects, and the expected action to the use of the trays generated by the Memoflex plates, are the sole responsibility of the dentist and/ or the professional responsible.

During the thermoforming process do not neglect the equipment. The material of the plates heats and softens quickly and can become unusable and still cause damage to the equipment if it softens too much.

Vacuum-sealed metalized packaging protects the plates from moisture and contaminants. The plates are highly hygroscopic (high moisture absorption capacity). After opening the vacuum packaging, plates that are not used immediately should remain stored in the same package to not absorb moisture and not have bubbles during heating in the thermoformed.

Thermoforming must be performed by trained people, who have knowledge of the process and thermoforming equipment.

## 16. Mode of Use:

- The mold must be fully cured and dry.
- Position the mold in the center of the thermoforming equipment.
- It is recommended to apply a medical release agent in the mold (to prevent the plate from sticking to the model).
- Turn on the thermoformed and set the appropriate temperature and heating time (see thermoforming parameters of the equipment, which vary according to the model and brand of the equipment).
- Remove the Memoflex plate from the packaging and position it in the thermoforming structure.
- Carry out the forming process according to the thermoforming equipment manufacturer's instructions.
- After thermoforming stop and finish the product.
- Finally, clean the product with a neutral detergent that meets medical specifications (clean it below 40°C).

### 16.1. Finishing:

- Use a saw disc or a helical drill to cut the product conformed to the Memoflex plate.
- Use a polishing brush on a high-speed motor to sand the product by removing all burr, applying light forces.

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**NOTES:**

- After opening the package, the plate should be formed within 15 minutes, after this period there is an increase in the probability of the occurrence of bubbles during thermoforming, due to the hygroscopic characteristic of the product.
- Heating time, temperature, and pressure vary between different machines.
- If the Memoflex board does not mold well to the mold, the heating time can be increased by 3-5 seconds at a time until the product is well-formed.
- If plaque sags, forming abnormal wrinkles or deformations, reduce heating time until no abnormal wrinkles or deformations occur.

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