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1. Product's name:

1.1. Technical Name: Orthodontic Bracket

1.2. Comercial Name: Aditek Orthodontic Brackets

1.3. Commercial Model: EasyClip+ Roth Self-Ligating Brackets; EasyClip+ MBT Self-Ligating Brackets; Andrews EasyClip+ Self-Ligating Brackets; EasyClip+ Damon Low Torque Self-Ligating Brackets; EasyClip+ Damon Standard Self-Ligating Brackets; EasyClip+ Damon Super Torque Self-Ligating Brackets; EasyClip R Roth Self-Ligating Brackets; EasyClip R MBT Self-Ligating Brackets; Andrews EasyClip R Self-Ligating Brackets; EasyClip R Damon Low Torque Self-Ligating Brackets; EasyClip R Damon Standard Self-Ligating Brackets; EasyClip R Damon Super Torque Self-Ligating Brackets; EasyClip Lock Roth Self-Ligating Brackets; EasyClip Lock MBT Self-Ligating Brackets; Andrews EasyClip Lock Self-Ligating Brackets; Damon Low Torque EasyClip Lock Self-Ligating Brackets; Damon Standard EasyClip Lock Self-Ligating Brackets; Damon Super Torque EasyClip Lock Self-Ligating Brackets; EasyClip Lock Brackets; EasyClip+ brackets; EasyClip Ramp Brackets; EasyClip+ passive brackets; EasyClip+ Interactive Brackets; EasyClip+ Hybrid Brackets; EasyClip+ Two-Dimensional Brackets; EasyClip+ Hybrid Two-Dimensional Brackets; Lotus Roth Self Ligating Bracket; Bracket Self Ligating Lotus MBT; Bracket Self Ligating Lotus Andrews; Bracket Self Ligating Lotus Damon Low Torque; Bracket Self Ligating Lotus Damon Standard; Bracket Self Ligating Lotus Damon High Torque; BioClip Roth Self-Ligating Brackets; BioClip MBT Self-Ligating Brackets; Andrews BioClip Self-Ligating Brackets; Damon Low Torque BioClip Self-Ligating Brackets; Damon Standard BioClip Self-Ligating Brackets; Damon Super Torque BioClip Self-Ligating Brackets; BioClip Voudouris Self-Ligating Brackets; BioClip PI Roth Self-Ligating Brackets; BioClip PI MBT Self-Ligating Brackets; Andrews BioClip PI Self-Ligating Brackets; BioClip PI Damon Low Torque Self-Ligating Brackets; Damon Standard BioClip PI Self-Ligating Brackets; BioClip PI Damon Super Torque Self-Ligating Brackets; BioClip PI Voudouris Self-Ligating Brackets; Self-Ligating BioClip Two-Dimensional Dual Slot Brackets; Hybrid Two-Dimensional BioClip Self-Ligating Brackets; BioClip PI Two-Dimensional Dual Slot Self-Ligating Brackets; Hybrid Two-Dimensional BioClip PI Self-Ligating Brackets; Select Roth Self-Ligating Brackets; Select MBT Self-Ligating Brackets; Select Damon Low Torque Self-Ligating Brackets; Select Damon Standard Self-Ligating Brackets; Select Damon Super Torque Self-Ligating Brackets; Select Standard I Self-Ligating Brackets; Select Standard II Self-Ligating Brackets; Select Standard III Self-Ligating Brackets; Evolution Standard I Self-Ligating Brackets; Evolution Standard II Self-Ligating Brackets; Evolution Standard III Self-Ligating Brackets; Variflex Damon Low Torque Self-Ligating Brackets; Variflex Damon Standard Self-Ligating Brackets; Variflex Damon Super Torque Self-Ligating Brackets; Variflex Roth Self-Ligating Brackets; Variflex MBT Self-Ligating Brackets; Stilo 3D Lingual Self-Ligating Brackets; Slide H5 Roth Self-Ligating Brackets; Self-Ligating Slide H5 MBT Brackets; Damon Low Torque H5 Slide Self-Ligating Brackets; Damon Standard Slide H5 Self-Ligating Brackets; Damon Super Torque H5 Slide Self-Ligating Brackets; Affinity Armando H Self-Ligating Brackets; Affinity Armando M Self-Ligating Brackets; Affinity Armando L Self-Ligating Brackets; Affinity Armando HM Self-Ligating Brackets; Affinity Armando HL Self-Ligating Brackets; Armando MH ffinity self-ligating brackets; Affinity Armando ML Self-Ligating Brackets; Affinity Armando LH Self-Ligating Brackets; Armando Affinity Self-Ligating Brackets.

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www.siq.si

4. Indication of Use:

Self-ligating brackets are indicated as auxiliary components in orthodontic treatment.

The purpose of the use of the brackets is to serve as a support and fitting point for wires, preformed archwires, or other auxiliary products used in conventional orthodontic treatments.

The products were developed to be used in the oral cavity environment, directly on the teeth, to support orthodontic treatments in humans, promoting a better positioning of the teeth within the dental arches, seeking a balanced occlusion.

Brackets are used in conjunction with archwires, wires, springs, bands, elastic bands, and other auxiliary products necessary for orthodontic treatment.

Self-ligating brackets have a clip that keeps the wire inside the slot, eliminating the use of ligatures or tie wires.

They are indicated to treat patients who have malocclusion problems such as prognathism, retrognathism, deep bite, open bite, crowding, anterior crossbite, posterior crossbite, and diastemas.

They are chosen by orthodontic professionals who seek faster clinical care in the replacement of orthodontic wires or archwires.

NOTE: The choice of material to be used is made by the qualified professional, based on their knowledge acquired during their training, together with the diagnosis and treatment plan for each patient.

5. Users:

The application of self-ligating brackets is restricted to dental professionals. Use by people outside this context may result in harm to the patient.

6. Performance Characteristics:

As the brackets are glued directly to the surface of the teeth, all the force applied to it is transferred directly to the tooth, causing the position of the tooth to be changed and consequently enabling its correction to the correct desired position.

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Usually, an orthodontic treatment without interruption takes from 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion, and oral and systemic conditions of the patient, and may extend according to the complexity of the case.

7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient chewing function facilitating swallowing, better digestion, prevention of occlusal problems, better speech, and breathing.

8. Warnings:

Product containing Chromium and Nickel.

Aditek's products are designed and manufactured for single use.

If the packaging has been tampered with, do not use the product.

Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

The product is not recommended in the MRI setting.

The product is safe for use in pregnant women.

The composition of the product does not require specific procedures for its transport and does not present risks to people.

The product is supplied in the non-sterile condition.

The professional must use good clinical practices for the disposal of medical products in accordance with current health regulations.

Aditek do Brasil S.A. is not responsible for the results obtained by the application of the product without due precautions or non-observance of warnings and instructions for use.

In the event of serious incidents with the product, Aditek do Brasil S.A. must be reported to the competent authority in the national scope in which the dentist is established.

9. Contraindications:

- Patients with poor oral hygiene,
- Patients unable to cooperate with treatment,
- Patients who are allergic to chromium and/or nickel,
- Disease or limitation that can make treatment difficult,
- Existing bone or root resorption,
- Decalcification of existing tooth enamel,
- Existing periodontal complications.

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NOTE: Aditek do Brasil S.A. supplies products only to dental professionals. It is the sole responsibility of the dentist to identify and/or possible contraindications that may prevent the use of the product.

10. Precautions:

The product may cause allergic reactions in patients susceptible to Chromium and Nickel allergy. In case of a reaction, remove the device and advise the patient to seek medical attention.

The reuse or reprocessing of the product can cause cross-infection, decreased bonding efficiency, and loss of mechanical properties due to natural wear and forces applied in the removal of the product.

Avoid risks related to the transmission of microorganisms or possible counterfeiting by using only products that have their packaging sealed.

The product should be applied by a dentist who should follow the instructions for use and guidelines specified by the manufacturer.

In the MRI environment, the product will reflect the magnetic waves generated by the CT scanner, preventing, or masking the correct reading of the scan, so they should always be removed before MRIs of the head and neck.

Due to hormonal changes during pregnancy, a woman is more susceptible to periodontal problems. During orthodontic treatment, the difficulty of cleaning can aggravate these problems. The patient must be referred for professional cleaning at regular intervals.

The procedure of bonding and removing the brackets on the teeth must be performed according to the manufacturer's instructions for the use of the orthodontic adhesive purchased.

The product is supplied clean in a non-sterile condition, and it is at the discretion of the professional to perform the sterilization.

18. Residual Risks:

All finished orthodontic treatments are subject to recurrence. After treatment, control should be determined by the professional to avoid or minimize recurrence.

11. Undesirable side effects:

The use of the medical device can lead to discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort and pain.

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12. Composition:

AISI 17-4 PH Stainless Steel – Bracket								
Chemical composition limits (%)								
C	Mn	P	S	Si	Cr	Ni	Cu	Fe
≤ 0.070	≤ 1.00	≤ 0.040	≤ 0.030	≤ 1.00	15.0 to 17.5	3.0 to 5.0	3.0 to 5.0	Remaining
ASTM: A564								

Nickel Titanium NiTi #1 - Clip									
Chemical composition limits (%)									
Ni	C	Co	Cu	H	Fe	Nb	N+O	Cr	Ti
54.5 to 57.0	≤ 0.050	≤ 0.050	≤ 0.010	≤ 0.005	≤ 0.050	≤ 0.025	≤ 0.050	≤ 0.010	Remaining
ASTM: F2063									

13. Validity:

10 years

14. Symbols and Meanings:

	Manufacturer		Authorized Representative in the European Community
	See instructions for use		Complies with European directive. XXXX represents the certifying body number
	Attention		Product contains chromium and nickel
	Do not use if package is damaged or tampered with		Do not reuse the product
	Product reference code		Batch number
	Expiration date		

15. Patient Information:

Products containing Chromium and Nickel in their composition can cause allergic reactions in people susceptible to these components.

The patient must perform good daily hygiene to avoid the installation of bacterial plaque and consequent periodontal diseases.

The pregnant patient must be referred for professional cleaning at regular intervals, due to the increased susceptibility to periodontal problems during pregnancy.

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It is important to take care when chewing certain types of foods, especially those that are harder or crunchier, as they can peel off or break the brackets. These foods should be cut before eating. Hard foods, such as popcorn and ice, should be avoided.

Do not put objects in your mouth, such as pens and pencils, and be careful not to hit or bite the brush frame when brushing your teeth, which may break or loosen the bracket when subjected to a lever.

In the event of a breakdown, seek a dentist.

In case of ingestion or aspiration, seek medical attention.

When participating in sports, always wear mouth guards, as impacts can damage the braces and injure the oral cavity.

The patient should follow the professional's instructions regarding restraint to avoid recurrence.

The product is not recommended in the MRI setting.

16. Mode of use:

The steps below should only be performed by dental professionals.

The brackets should be glued directly to the surface of the teeth at their base, with the use of orthodontic adhesives, following the adhesive manufacturer's instructions for use.

Place the instrument (explorer) in the clip hole, located below the slot, between the lower fins and perpendicular to the buccal surface of the tooth. Rotate the instrument gently to the right or left; If the piece has a hook, the rotation must be in the opposite direction to the hook.

The clip can also be opened with a fine-tipped explorer with a gentle downward lever movement, placing the instrument inside the hole. The lower bracket clips open to the gingival and the upper ones to the occlusal.

Position the selected arch according to the treatment to be performed. Using tweezers, place one end over the fins of the bracket and the other end on the clip.

Using tweezers, place one end over the tabs of the bracket and the other end into the clip.

Press the tweezers vertically. The first "click" unlocks the opening that was made in the clip. The second, stronger "click" will lock the clip, closing the slot.

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After the treatment, remove the brackets and orthodontic adhesive from the teeth using an appropriate instrument, according to the manufacturer's instructions for use of the orthodontic adhesive purchased.

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