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1. Product Name:

1.1. Technical Name: Orthodontic Brackets

1.2. Commercial Name: Aditek Orthodontic Brackets

1.3. Commercial Model: BioMax Edgewise Brackets; Roth BioMax Brackets; BioMax MBT Brackets; Andrews BioMax Brackets; BioMax Standard I Brackets; BioMax Standard II Brackets; BioMax Standard III Brackets; BioMax Ricketts Brackets; Jorge Rodrigues BioMax Brackets; BioMax Brackets; One Piece Brackets / Technique: Straight Wire / Prescription: Roth; One Piece Brackets / Technique: Straight Wire / Prescription: MBT; One Piece Brackets / Technique: Edgewise / Prescription: Edgewise Standard; One Piece Brackets; Vector Brackets / Technique: Straight Wire / Prescription: Capellozza Standard I; Vector Brackets / Technique: Straight Wire / Prescription: Capellozza Standard II; Vector Brackets / Technique: Straight Wire / Prescription: Capellozza Standard III; Vector Brackets / Technique: Straight Wire / Prescription: Andrews; Vector Brackets / Technique: Straight Wire / Prescription: Roth; Vertical Slot Vector Brackets / Technique: Straight Wire / Prescription: Roth; Vector Mini-Twin Brackets / Technique: Straight Wire / Prescription: Roth; Vector Brackets / Technique: Straight Wire / Prescription: MBT; Vector Brackets / Technique: Bioprogressive / Prescription: Ricketts; Vector Brackets / Technique: Edgewise / Prescription: Edgewise Standard; Vector S Edgewise Brackets; Vector S Roth brackets; Vector S MBT brackets; Brackets Vector S Andrews; Brackets Vector S Standard I; Vector S Standard II brackets; Vector S Standard III brackets; Vector S Ricketts Brackets; Brackets Vector Jorge Rodrigues; Brackets Vector Election; Simple Vector Brackets; Vector Bio XP Brackets; Vector Single Brackets; Vector Brackets; Two-dimensional vector brackets; Vector+ brackets; Two-dimensional Vector+ brackets; Edgewise Bionic Bracket; Roth Bionic Bracket; Bracket Bionic MBT; Bracket Bionic Andrews; Standard Bionic Bracket I; Standard Bionic Bracket II; Standard Bionic Bracket III; Bracket Bionic Ricketts; Brackets Tube for Bonding - 1st Convertible Double Rectangular Lower Molar With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Tube Brackets for Bonding - 1st Lower Molar Double PLA Convertible With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Mini Button Brackets for Bonding; Straight Hook Type Brackets (Rectangular Arch); Tube Type Brackets for fitting (Palatine Bar) with Window and Hook; Tube Type Brackets for fitting (Palatine Bar) with window and Latch without hook; Tube Type Brackets for fitting (Palatine Bar) with Window and Lock with Hook for Mesial or Distal Bar; Brackets Weld Tube - 1st Upper Molar Triple Occlusal Convertible With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Brackets Weld Tube - 1st Convertible Triple Occlusal Upper Molar With Hook / Technique: Bioprogressive / Prescription: Ricketts; Brackets Weld Tube - 1st Upper Molar Triple Occlusal Convertible With Hook / Technique: Straight Wire / Prescription: MBT; Brackets Weld Tube – 1st Upper Molar Double Gingival Convertible With Hook / Technique: Straight Wire / Prescription: Capellozza; Weld Tube Brackets – 1st Double Upper Molar; Tube Brackets for Welding- 1st Upper Molar Double Occlusal Convertible With Hook / Technique: Straight Wire / Prescription: MBT; Weld Tube Brackets - 1st Upper Molar Double Occlusal Convertible With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Tube Brackets for Welding- 1st Upper Molar Double Occlusal Convertible With Hook / Technique: Edgewise/Edgewise Standard; Brackets Weld Tube – 1st Upper Molar Double Non-Convertible With Hook / Technique: Bioprogressive / Prescription: Ricketts; Weld Tube Brackets – 1st Upper Molar Double Non-Convertible With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Tube Brackets for Welding- 1st Upper Molar Single Convertible With Hook / Technique: Straight Wire / Prescription: Capellozza; Tube Brackets for Welding- 1st Upper Molar

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Single Convertible With Hook / Technique: Straigh Wire / Prescription: Andrews/Roth; Tube Brackets for Welding- 1st Upper Molar Simple Convertible With Hook / Technique: Straight Wire / Prescription: MBT; Tube Brackets for Welding- 1st and 2nd Upper Molars Simple Non-Convertible With Hook / Technique: Straight Wire / Prescription: Capelozza; Weld Tube Brackets - 1st and 2nd Upper Molars Simple Non-Convertible With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Weld Tube Brackets - 1st and 2nd Single Upper and Lower Molars Non-Convertible With Hook / Technique: Edgewise / Prescription: Edgewise; Tube Brackets for Welding- 1st and 2nd Upper Molars Simple Non-Convertible With Hook / Technique: Straight Wire / Prescription: MBT; Brackets Tube for Welding -1st Upper and Lower Molars Simple Convertible with Hook / Technique: Edgewise / Prescription: Edgewise; Weld Tube Brackets - 1st Convertible Double Rectangular Lower Molar with Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Brackets Weld Tube - 1st Lower Molar Double PLA Convertible with Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Weld Tube Brackets – 1st Lower Molar Double PLA Convertible with Hook / Technique: Straight Wire / Prescription: MBT; Brackets Weld Tube - 1st Lower Molar Double Non-Convertible with Hook / Technique: Bioprogressive / Prescription: Ricketts; Weld Tube Brackets - 1st Convertible Single Lower Molar with Hook / Technique: Straight Wire / Andrews/Roth Prescription; Tube Brackets For Welding- 1st Lower Molar Simple Convertible with Hook / Technique: Straight Wire / Prescription: MBT; Weld Tube Brackets - 1st and 2nd Lower Molars Simple Non-Convertible with Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Weld Tube Brackets - 1st and 2nd Lower Molars Simple Non-Convertible with Hook / Technique: Straight Wire / Prescription: MBT; Tube Brackets for Bonding - 1st Convertible Single Upper Molar With Hook / Technique: Straight Wire / Prescription: Capelozza; Tube Brackets for Bonding - 1st Convertible Single Upper Molar With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Tube Brackets for Bonding- 1st Upper Molar Simple Convertible With Hook / Technique: Straight Wire / Prescription: MBT; Tube Brackets for Bonding - 1st and 2nd Single Non-Convertible Upper Molars With Hook / Technique: Straight Wire / Andrews / Roth Prescription; Tube Brackets for Bonding - 1st and 2nd Single Non-Convertible Upper Molars With Hook / Technique: Straight Wire / Prescription: Capelozza; Tube Brackets for Bonding - 1st and 2nd Maxillary Molars Simple Non-Convertible With Hook / Technique: Straight / Prescription: MBT; Tube Brackets for Bonding - 1st and 2nd Single Upper and Lower Molars Non-Convertible with Hook / Technique: Edgewise / Prescription: Edgewise; Brackets Tube for Bonding - 1st Lower Molar Double Non-Convertible With Hook / Technique: Bioprogressive / Prescription: Ricketts; Tube Brackets for Bonding - 1st Convertible Single Mandibular Molar With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Brackets Tube Rim Bonding - 1st Lower Molar Simple Convertible with Hook / Technique: Straight Wire / Prescription: MBT; Tube Brackets for Bonding - 1st and 2nd Lower Molars Simple Non-Convertible With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Tube Brackets for Bonding - 1st and 2nd Lower Molars Simple Non-Convertible With Hook / Technique: Straight Wire / Prescription: MBT; Tube Brackets for Bonding - 1st Single Upper and Lower Molars Convertible With Hook / Edgewise Technique / Prescription: Edgewise; Tube Brackets for Bonding - 1st Upper Molar Triple Occlusal Convertible With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Tube Brackets for Bonding - 1st Convertible Triple Occlusal Upper Molar With Hook / Technique: Straight Wire / Prescription: MBT; Tube Brackets for Bonding - 1st Convertible Triple Occlusal Maxillary Molar with Hook / Technique: Bioprogressive / Prescription: Ricketts; Tube Brackets for Bonding - 1st Upper Molar Double Gingival Convertible with Hook / Technique: Straight Wire / Prescription: Capelozza; Tube Brackets for Bonding - 1st Convertible Double

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Occlusal Upper Molar With Hook / Technique: Straight Wire / Prescription: MBT; Tube Brackets for Bonding - 1st Upper Molar Double Gingival Convertible With Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Tube Brackets for Bonding - 1st Convertible Double Occlusal Maxillary Molar with Hook / Technique: Straight Wire / Prescription: Andrews/Roth; Tube Brackets for Bonding - 1st Convertible Double Occlusal Upper Molar With Hook / Technique: Edgewise / Prescription: Edgewise; Tube Brackets for Bonding - 1st Upper Molar Double Non-Convertible With Hook / Technique: Straight Wire / Andrews/Roth Prescription; Tube Brackets for Bonding - 1st Lower Molar Double PLA Convertible With Hook / Technique: Straight Wire / Prescription: MBT; Tube Brackets for Bonding - 1st Upper Molar Double Non-Convertible With Hook / Technique: Bioprogressive / Ricketts Prescription; Roth Synergy Brackets; MBT Synergy Brackets; Ricketts Synergy Brackets; Standard Synergy Bracket I; Standard Synergy II Brackets; Standard Synergy Brackets III.

1.4. Anvisa Notification: 10331430016

2. Manufacturer:

Aditek do Brasil S.A.
 Address: Cesário Mota street, 14.
 Zip Code: 14140-000 - Cravinhos – SP - Brazil
 Phone: +55 (16) 3951-9355 / 0800-0159520
 CNPJ 64.602.097/0001-95
 Technical Manager: José Vicente Junior - CREA: 5069373070
 Site: www.aditek.com.br

3. Certifying Body:

SIQ Ljubljana
 SLO-1000 Ljubljana, Trzaska cesta 2
 Phone: +386 1 4778 100
info@siq.si
www.siq.si

4. Indication for Use:

Conventional Metal Brackets are indicated as auxiliary components in orthodontic treatment. The purpose of the use of the brackets is to serve as a support and fitting point for wires, preformed archwires, or other auxiliary products used in conventional orthodontic treatments.

The products were developed to be used in the oral cavity environment, directly on the teeth, to support orthodontic treatments in humans, promoting a better positioning of the teeth within the dental arches, seeking a balanced occlusion.

Brackets are used in conjunction with archwires, wires, springs, bands, elastic bands, and other auxiliary products necessary for orthodontic treatment.

Conventional brackets must be fitted with individual elastic or metal bandages to keep the wire inside the slot.

They are indicated to treat patients who have malocclusion problems such as prognathism, retrognathism, deep bite, open bite, crowding, anterior crossbite, posterior crossbite, and diastemas.

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NOTE: The choice of the material to be used is made by the qualified professional, based on his/her knowledge acquired during his/her training together with the diagnosis and treatment plan for each patient.

5. Users:

The application of orthodontic brackets is restricted to dental professionals. Use by people outside this context may result in harm to the patient.

6. Performance Characteristics:

As the brackets are glued directly to the surface of the teeth, all the force applied to it is transferred directly to the tooth, causing the position of the tooth to be changed and consequently enabling its correction to the correct desired position.

Usually, an orthodontic treatment without interruption takes from 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion, and oral and systemic conditions of the patient, and may extend according to the complexity of the case.

7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient chewing function facilitating swallowing, better digestion, prevention of occlusal problems, better speech, and breathing.

8. Warnings:

Product containing Chromium and Nickel.

Aditek's products are designed and manufactured for single use.

If the packaging has been tampered with, do not use the product.

Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

The product is not recommended in the MRI setting.

The product is safe for use in pregnant women.

The composition of the product does not require specific procedures for its transport and does not present risks to people.

The product is supplied in the non-sterile condition.

The professional must use good clinical practices for the disposal of medical products in accordance with current health regulations.

Aditek do Brasil S.A. is not responsible for the results obtained by the application of the product without due precautions or non-observance of warnings and instructions for use.

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In the event of serious incidents with the product, Aditek do Brasil S.A. must notify the competent authority at the national level in which the dentist is established.

9. Contraindications:

- Patients with poor oral hygiene,
- Patients with an inability to cooperate with treatment,
- Patients who have an allergy to chromium and/or nickel,
- Illness or limitation that may hinder treatment,
- Existing bone or root resorption,
- Descaling of existing tooth enamel,
- Existing periodontal complications.

NOTE: Aditek do Brasil S.A. supplies products only to dental professionals. It is the sole responsibility of the dentist to identify and/or possible contraindications that may prevent the use of the product.

10. Precautions:

The product may cause allergic reactions in patients susceptible to Chromium and Nickel allergy. In case of reaction, remove the device and advise the patient to seek medical attention.

The reuse or reprocessing of the product can cause cross-infection, decreased bonding efficiency, and loss of mechanical properties due to natural wear and forces applied in the removal of the product.

Avoid risks related to the transmission of microorganisms or possible counterfeiting by using only products that have their packaging sealed.

The product should be applied by a dentist who should follow the instructions for use and guidelines specified by the manufacturer.

In the MRI environment, the product will reflect the magnetic waves generated by the CT scanner, preventing, or masking the correct reading of the scan, so they should always be removed before MRIs of the head and neck.

Due to hormonal changes during pregnancy, a woman is more susceptible to periodontal problems. During orthodontic treatment, the difficulty of cleaning can aggravate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

The procedure of bonding and removing the brackets on the teeth must be performed according to the manufacturer's instructions for use of the orthodontic adhesive purchased.

The product is supplied clean in a non-sterile condition, and it is at the discretion of the professional to perform the sterilization.

11. Residual Risks:

All finished orthodontic treatments are subject to recurrence. After treatment, control should be determined by the professional to avoid or minimize recurrence.

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12. Undesirable side effects:

The use of the medical device can lead to discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort and pain.

13. Composition:

Stainless Steel AISI 17-4 PH								
Chemical Composition Limits (%)								
C	Mn	P	S	Oneself	Cr	Nu	Ass	Fe
≤ 0,070	≤ 1,00	≤ 0,040	≤ 0,030	≤ 1,00	15.0 to 17.5	3.0 to 5.0	3.0 to 5.0	Remainder
ASTM: A564								

14. Validity:

10 years

15. Symbols and Meanings:

	Manufacturer		Authorized representative in the European Community
	See instructions for use		Complies with the European directive. XXXX represents the number of the certifying body
	Attention		Product contains chromium and nickel
	Do not use if package is damaged or tampered with		Do not reuse the product
	Product Reference Code		Batch number
	Expiration date		

16. Patient Information:

Product containing Chromium and Nickel in its composition, which can cause allergic reactions in people susceptible to these components.

The patient must perform good daily hygiene in order to avoid the installation of bacterial plaque and consequent periodontal diseases.

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It is important that the pregnant patient is referred for professional cleaning at regular intervals, due to the increased susceptibility to periodontal problems during pregnancy.

It is important to take care when chewing certain types of foods, especially those that are harder or crunchier, as they can peel off or break the brackets. These foods should be cut before eating. Hard foods, such as popcorn and ice, should be avoided.

Do not put objects in your mouth, such as pens and pencils, and be careful not to hit or bite the brush frame when brushing your teeth, which may break or loosen the bracket when subjected to a lever.

In the event of a breakdown, seek a dentist.

In case of ingestion or aspiration, seek medical attention.

When participating in sports, always wear mouth guards, as impacts can damage the braces and injure the oral cavity.

The patient should follow the professional's instructions regarding restraint to avoid recurrence.

The product is not recommended in the MRI setting.

17. Mode of Use:

The steps below should only be performed by dental professionals.

- Prepare the surface of the teeth according to the manufacturer's instructions for use of the orthodontic adhesive purchased;
- Glue the brackets to the teeth according to the manufacturer's instructions for use of the orthodontic adhesive purchased;
- Position the selected arch according to the treatment to be performed;
- Tie the bow with elastic or metal bandage.

After the treatment, remove the brackets and orthodontic adhesive from the teeth using an appropriate instrument, according to the manufacturer's instructions for use of the orthodontic adhesive purchased.

NOTE: Orthodontic Brackets should be bonded directly to the surface of the teeth at their base, with the use of orthodontic adhesives.

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