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1. Product Name:

1.1. Technical Name: Orthodontic Bands

1.2. Commercial Name: Aditek Orthodontic Bands

1.3. Commercial Model: Right Lower Molar Band: LR28, LR28.5, LR29, LR29.5, LR30, LR30.5, LR31, LR31.5, LR32, LR32.5, LR33, LR33.5, LR34, LR34.5, LR35, LR35.5, LR36, LR36.5, LR37, LR37.5, LR38, LR38.5, LR39, LR39.5, LR40, LR40.5, LR41, LR41.5, LR42, LR42.5, LR43, LR43.5, LR44. Left Lower Molar Band: LL28, LL28.5, LL29, LL29.5, LL30, LL30.5, LL31, LL31.5, LL32, LL32.5, LL33, LL33.5, LL34, LL34.5, LL35, LL35.5, LL36, LL36.5, LL37, LL37.5, LL38, LL38.5, LL39, LL39.5, LL40, LL40.5, LL41, LL41.5, LL42, LL42.5, LL43, LL43.5, LL44. Right Upper Molar Band: UR28, UR28.5, UR29, UR29.5, UR30, UR30.5, UR31, UR31.5, UR32, UR32.5, UR33, UR33.5, UR34, UR34.5, UR35, UR35.5, UR36, UR36.5, UR37, UR37.5, UR38, UR38.5, UR39, UR39.5, UR40, UR40.5, UR41, UR41.5, UR42, UR42.5, UR43, UR43.5, UR44. Upper Left Molar Band: UL28, UL28.5, UL29, UL29.5, UL30, UL30.5, UL31, UL31.5, UL32, UL32.5, UL33, UL33.5, UL34, UL34.5, UL35, UL35.5, UL36, UL36.5, UL37, UL37.5, UL38, UL38.5, UL39, UL39.5, UL40, UL40.5, UL41, UL41.5, UL42, UL42.5, UL43, UL43.5, UL44

1.4. Anvisa Notification: 10331430013

2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Mota street, 14.

Zip Code: 14140-000 - Cravinhos – SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Manager: José Vicente Junior - CREA: 5069373070

Site: www.aditek.com.br

3. Certifying Body:

SIQ Ljubljana

SLO-1000 Ljubljana, Trzaska cesta 2

Phone: +386 1 4778 100

info@siq.si

www.siq.si

4. Indication for Use:

The application of Orthodontic Bands is restricted to dental professionals with qualification in Orthodontics. Use by people outside this context may result in irreversible damage to the patient.

The products were developed to be used in the oral cavity environment, directly on the teeth, to support orthodontic treatments in humans, promoting a better positioning of the teeth within the dental arches towards a correct position and correct occlusion.

The bands are used in conjunction with brackets, archwires, wires, springs, elastics, and other auxiliary products necessary for orthodontic treatment.

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They are indicated to treat patients who have malocclusion problems such as prognathism, retrognathism, deep bite, open bite, crowding, anterior crossbite, posterior crossbite, and diastemas.

NOTE: The choice of the material to be used is made by the qualified professional, based on his/her knowledge acquired during his/her training, together with the diagnosis and treatment plan for each patient.

5. Users:

The application of orthodontic bands is restricted to dental professionals with qualification in Orthodontics. Use by people outside this context may result in irreversible damage to the patient.

6. Performance Characteristics:

Usually, an orthodontic treatment without interruption takes from 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion, and oral and systemic conditions of the patient, and may extend according to the complexity of the case.

7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient chewing function facilitating swallowing, better digestion, prevention of occlusal problems, better speech, and breathing.

8. Warnings:

Product containing Chromium and Nickel.

Aditek's products are designed and manufactured for single use.

If the packaging has been tampered with, do not use the product.

Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

The product is not recommended in the MRI setting.

The product is safe for use in pregnant women.

The composition of the product does not require specific procedures for its transport and does not present risks to people.

The product is supplied in the non-sterile condition.

The bands used in the selection of the best size, which have been placed in the patient's mouth, before returning to the box should be sterilized by undergoing an ultrasound bath and autoclaved at 134°C for at least 30 minutes.

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The professional must use good clinical practices for the disposal of medical products in accordance with current health regulations.

Aditek do Brasil S.A. is not responsible for the results obtained by applying the product without due precautions or non-observance of warnings and instructions for use.

In the event of serious incidents with the product, Aditek do Brasil S.A. must notify the competent authority at the national level in which the dentist is established.

9. Contraindications:

- Patients with poor oral hygiene,
- Patients with an inability to cooperate with treatment,
- Patients who have an allergy to chromium and/or nickel,
- Illness or limitation that may hinder treatment,
- Existing bone or root resorption,
- Descaling of existing tooth enamel,
- Existing periodontal complications.

It is the sole responsibility of the orthodontist to identify and/or possible contraindications that may prevent the use of the product.

10. Precautions:

The product may cause allergic reactions in patients susceptible to Chromium and Nickel allergy. In case of reaction, remove the device and advise the patient to seek medical attention.

Reuse or reprocessing of the product can cause cross-infection, decreased bonding efficiency, and loss of mechanical properties due to natural wear and forces applied in product removal.

Avoid risks related to the transmission of microorganisms or possible counterfeiting by using only products that have their packaging sealed.

The product should be applied by a dentist who should follow the instructions for use and guidelines specified by the manufacturer.

In the MRI environment, the product will reflect the magnetic waves generated by the CT scanner, preventing, or masking the correct reading of the scan, so they should always be removed before MRIs of the head and neck.

Due to hormonal changes during pregnancy, a woman is more susceptible to having periodontal problems. During orthodontic treatment, the difficulty of cleaning can aggravate these problems. It is important that the patient is referred for professional cleaning at regular intervals.

The procedure of bonding and removing the bands on the teeth must be performed according to the manufacturer's instructions for use of the orthodontic adhesive purchased.

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The product is supplied clean in a non-sterile condition, and it is at the discretion of the professional to perform the sterilization.

11. Residual Risks:

All finished orthodontic treatments are subject to recurrence. After treatment, control should be determined by the professional to avoid or minimize recurrence.

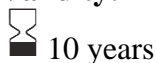
12. Undesirable side effects:

The use of the medical device can lead to discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort and pain.

13. Composition:

AISI 304L PH Stainless Steel Tape - Band								
Chemical Composition Limits (%)								
C	Oneself	Mn	P	S	Cr	Nu	N	Fe
≤ 0,030	≤ 0,75	≤ 2,00	≤ 0,045	≤ 0,030	18.0 to 20.0	8.0 to 12.0	≤ 0,10	Remainder
ASTM: A240								

14. Validity:

10 years

15. Symbols and Meanings:

	Manufacturer		Authorized representative in the European Community
	See instructions for use		Complies with the European directive. XXXX represents the number of the certifying body
	Attention		Product contains chromium and nickel
	Do not use if package is damaged or tampered with		Do not reuse the product
	Product Reference Code		Batch number
	Expiration date		

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16. Information to be given to the Patient:

Product containing Chromium and Nickel in its composition, which may cause allergic reactions in people susceptible to these components;

Perform good daily hygiene in order to avoid the installation of bacterial plaque and consequent periodontal diseases;

It is important that the pregnant patient is referred for professional cleaning at regular intervals, due to the increased susceptibility to periodontal problems during pregnancy;

Chewing hard foods can cause appliances to break or fall off;

In case of breakdown, seek an orthodontist;

In case of ingestion or aspiration, seek medical attention;

When participating in sports, always wear mouth guards, as impacts can damage the devices and injure the oral cavity;

Mention any undesirable side effects;

Mention about the risk of recurrence after treatment;

The patient should follow the professional's instructions regarding restraint to avoid recurrence;

The product is not recommended in the MRI setting.

17. Mode of Use:

The steps below should only be performed by dental professionals with a qualification in Orthodontics.

- Select the size of the band, choosing the one that best suits the tooth.
- Weld the accessories to be used,
- Prepare the surface of the teeth according to the manufacturer's instructions for use of the adhesive or orthodontic cement purchased.
- Apply the orthodontic resin or cement inside the band, according to the manufacturer's instructions for use of the adhesive or cement purchased,
- Position the band on the teeth using specific instruments, according to the manufacturer's instructions for use.

After treatment, remove the bands and the adhesive or orthodontic cement from the teeth, using an appropriate instrument, according to the manufacturer's instructions for use.

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1.3. Modelo comercial: Banda molar inferior derecha LR28, LR28.5, LR29, LR29.5, LR30, LR30.5, LR31, LR31.5, LR32, LR32.5, LR33, LR33.5, LR34, LR34.5, LR35, LR35.5, LR36, LR36.5, LR37, LR37.5, LR38, LR38.5, LR39, LR39.5, LR40, LR40.5, LR41, LR41.5, LR42, LR42.5, LR43, LR43.5, LR44. Banda molar inferior izquierda: LL28, LL28.5, LL29, LL29.5, LL30, LL30.5, LL31, LL31.5, LL32, LL32.5, LL33, LL33.5, LL34, LL34.5, LL35, LL35.5, LL36, LL36.5, LL37, LL37.5, LL38, LL38.5, LL39, LL39.5, LL40, LL40.5, LL41, LL41.5, LL42, LL42.5, LL43, LL43.5, LL44. Banda molar superior derecha: UR28, UR28.5, UR29, UR29.5, UR30, UR30.5, UR31, UR31.5, UR32, UR32.5, UR33, UR33.5, UR34, UR34.5, UR35, UR35.5, UR36, UR36.5, UR37, UR37.5, UR38, UR38.5, UR39, UR39.5, UR40, UR40.5, UR41, UR41.5, UR42, UR42.5, UR43, UR43.5, UR44. Banda molar superior izquierda: UL28, UL28.5, UL29, UL29.5, UL30, UL30.5, UL31, UL31.5, UL32, UL32.5, UL33, UL33.5, UL34, UL34.5, UL35, UL35.5, UL36, UL36.5, UL37, UL37.5, UL38, UL38.5, UL39, UL39.5, UL40, UL40.5, UL41, UL41.5, UL42, UL42.5, UL43, UL43.5, UL44

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