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## 1. Product Name:

**1.1. Technical Name:** Orthodontic Bracket

**1.2. Commercial Name:** Aditek Orthodontic Stop

**1.3. Commercial Model:** closed stop - 2 mm - round wire, closed stop - 3 mm - round wire, closed stop - 2 mm - rectangular wire, open stop.

**1.4. Anvisa Notification:** 10331430016

## 2. Manufacturer:

Aditek do Brasil S.A.

Address: Cesário Mota, 14.

Zip Code: 14140-000 - Cravinhos – SP - Brazil

Phone: +55 (16) 3951-9355 / 0800-0159520

CNPJ 64.602.097/0001-95

Technical Manager: José Vicente Junior - CREA: 5069373070

Website: [www.aditek.com.br](http://www.aditek.com.br)

## 3. Certifying Body:

SIQ Ljubljana

SLO-1000 Ljubljana, Trzaska cesta 2

Phone: +386 1 4778 100

[info@siq.si](mailto:info@siq.si)

[www.siq.si](http://www.siq.si)

## 4. Indication for Use:

The stops are indicated to prevent the displacement of the wire or arch between the slots of the brackets and as an aid in the placement of orthodontic springs.

The products were developed to be used in the oral cavity environment, with the aim of supporting orthodontic treatments in humans.

They are divided into two types, open stop (with rectangular interior) and closed stop (with round interior). The stops for round wires fit up to .020" wire, the stops for rectangular wires fit up to .021"x.025" wire, and for larger gauge wires it is recommended to use the open stop.

**NOTE:** The choice of the material to be used is made by the qualified professional, based on his/her knowledge acquired during his/her training together with the diagnosis and treatment plan for each patient.

## 5. Users:

The application is restricted to dental professionals. Use by people outside this context may cause harm to the patient.

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## 6. Performance Characteristics:

Usually, an orthodontic treatment without interruption takes from 1 to 3 years, varying in each case according to the facial pattern, type of malocclusion, and oral and systemic conditions of the patient, and may extend according to the complexity of the case.

## 7. Clinical Benefits:

The main benefits that these systems bring to the patient's life are aesthetic improvement with the alignment and leveling of the teeth, better hygiene, more efficient chewing function facilitating swallowing, better digestion, prevention of occlusal problems, better speech and breathing.

## 8. Warnings:

Product containing Chromium and Nickel.

Aditek's products are designed and manufactured for single use.

If the packaging has been tampered with, do not use the product.

Store the product in a cool, dry place away from sunlight, keeping it in its own packaging until use.

The product is not recommended in the MRI setting.

The product is safe for use in pregnant women.

The composition of the product does not require specific procedures for its transport and does not present risks to people.

The product is supplied in the non-sterile condition.

The professional must use good clinical practices for the disposal of medical products in accordance with current health regulations.

Aditek do Brasil S.A. is not responsible for the results obtained by the application of the product without due precautions or non-observance of warnings and instructions for use.

In the event of serious incidents with the product, Aditek do Brasil S.A. must be reported to the competent authority in the national scope in which the dentist is established.

## 9. Contraindications:

- Patients with poor oral hygiene,
- Patients with an inability to cooperate with treatment,
- Patients who have an allergy to chromium and/or nickel,
- Illness or limitation that may hinder treatment,

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- Existing bone or root resorption,
- Descaling of existing tooth enamel,
- Existing periodontal complications.

**NOTE:** Aditek do Brasil S.A. supplies products only to dental professionals. It is the sole responsibility of the dentist to identify and/or possible contraindications that may prevent the use of the product.

#### 10. Precautions:

The product may cause allergic reactions in patients susceptible to Chromium and Nickel allergy. In case of a reaction, remove the device and advise the patient to seek medical attention.

The reuse or reprocessing of the product can cause cross-infection, decreased bonding efficiency, and loss of mechanical properties due to natural wear and forces applied in the removal of the product.

Avoid risks related to the transmission of microorganisms or possible counterfeiting by using only products that have their packaging sealed.

The product should be applied by a dentist who should follow the instructions and guidelines specified by the manufacturer.

In the MRI environment, the product will reflect the magnetic waves generated by the CT scanner, preventing, or masking the correct reading of the scan, so they should always be removed before MRIs of the head and neck.

Due to hormonal changes during pregnancy, a woman is more susceptible to periodontal problems. During orthodontic treatment, the difficulty of cleaning can aggravate these problems. The patient must be referred for professional cleaning at regular intervals.

The product is supplied clean in a non-sterile condition, and it is at the discretion of the professional to perform the sterilization.

#### 11. Residual Risks:

All finished orthodontic treatments are subject to recurrence. After treatment, control should be determined by the professional to avoid or minimize recurrence.

#### 12. Undesirable Side Effects:

The use of the medical device can lead to discoloration or decalcification of teeth, bone and root resorption, periodontal complications, allergic reactions, difficulties in maintaining oral hygiene, damage to the oral mucosa and tongue, difficulty speaking or chewing, discomfort and pain.

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### 13. Composition:

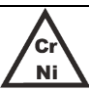
AISI 304L Stainless Steel Pipe – Closed Stop								
Chemical Composition Limits (%)								
C	Oneself	Mn	P	S	Cr	Nu	N	Fe
≤ 0.030	≤ 0.75	≤ 2.00	≤ 0.045	≤ 0.030	18.0 to 20.0	8.0 to 12.0	≤ 0.10	Remainder
<b>ASTM: A240</b>								

AISI 17-4 PH Stainless Steel – Stop Open								
Chemical Composition Limits (%)								
C	Mn	P	S	Oneself	Cr	Nu	Ass	Fe
≤ 0.070	≤ 1.00	≤ 0.040	≤ 0.030	≤ 1.00	15.0 to 17.5	3.0 to 5.0	3.0 to 5.0	Remainder
<b>ASTM: A564</b>								

### 14. Validity:

 10 years.

### 15. Symbols and Meanings:

	Manufacturer		Authorized representative in the European Community
	See instructions for use		Complies with the European directive. XXXX represents the number of the certifying body
	Attention		Product contains chromium and nickel
	Do not use if package is damaged or tampered with		Do not reuse the product
	Product Reference Code		Batch number
	Expiration date		

### 16. Patient Information:

The patient must perform good daily hygiene in order to avoid the installation of bacterial plaque and consequent periodontal diseases.

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It is important that the pregnant patient is referred for professional cleaning at regular intervals, due to the increased susceptibility to periodontal problems during pregnancy.

It is important to be careful when chewing certain types of foods, especially harder or crunchy ones, as they can damage braces. These foods should be cut before eating. Hard foods, such as popcorn and ice, should be avoided.

In the event of a breakdown, seek a dentist.

In case of ingestion or aspiration, seek medical attention.

When participating in sports, always wear mouth guards, as impacts can damage the braces and injure the oral cavity.

The patient should follow the professional's instructions regarding restraint to avoid recurrence.

The product is not recommended in the MRI setting.

#### **17. Mode of Use:**

The steps below should only be performed by orthodontic professionals.

Mark the desired position in which the stop will be fixed on the archwire or orthodontic wire.

Adjust the stop to the marked position.

With the help of suitable pliers, crimp the stop on the archwire or orthodontic wire, the compression deformation will cause the stop to be locked on the archwire or orthodontic wire.

**NOTE:** Always use appropriate equipment during fastening, in order to prevent the occurrence of unwanted damage to the wire or arch, or the displacement of the stop from its attachment point.

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